

GMP
ISO9001
ISO14001
ISO13485
CERTIFIED

PIHSIANG®

SHOPRIDER®

MODEL: **TE-889SL**

(PATENTED)

PIHSIANG MACHINERY MFG. CO., LTD.

*NO.108, HSIN-HE ROAD, HSIN-FENG HSIANG 304
HSINCHU COUNTY, TAIWAN*

MODEL :


[illegible]

<FRONT PAGE=
TE-889SL>

TABLE OF CONTENTS
ILLUSTRATION


<NOTE FOR USE>

INDEX 1	FRONT CHASSIS ASSEMBLY-----	前車架總成-----	1-1~2
INDEX 2	REAR CHASSIS ASSEMBLY-----	後車架總成-----	2-1~3
INDEX 3	DRIVE-TRAIN ASSEMBLY-----	馬達剎車變速箱總成-----	3-1
INDEX 4	MOTOR ASSEMBLY-----	馬達總成-----	4-1
INDEX 5	REAR WHEEL ASSEMBLY-----	後輪總成-----	5-1
INDEX 6	FRONT AXLE ASSEMBLY-----	導向輪總成-----	6-1~3
INDEX 7	CONNECTING ROD ASSEMBLY-----	導向連桿總成-----	7-1
INDEX 8	FRONT WHEEL ASSEMBLY-----	前輪總成-----	8-1
INDEX 9	HEAD LIGHT ASSEMBLY-----	前燈總成-----	9-1
INDEX 10	LOOP HANDLE BAR-----	環把披覆+固定座-----	10-1
INDEX 11	HANDLE BAR JOINT BASE ASSEMBLY-----	把手連接座總成-----	11-1
INDEX 12	DELUXE SEAT ASSEMBLY-----	大折疊椅模組+滑道-----	12-1
INDEX 13	FRONT CHASSIS COVER ASSEMBLY-----	前蓋板總成-----	13-1
INDEX 14	REAR CHASSIS COVER ASSEMBLY-----	後蓋板總成-----	14-1
INDEX 15	HANDLE BAR COVER ASSEMBLY-----	把手蓋總成-----	15-1~2
INDEX 16	UPPER CONTROL ASSEMBLY-----	上控制總成-----	16-1~2
INDEX 17	HANDLE BAR ASSEMBLY-----	二合一環把總成(大)B-----	17-1
INDEX 18	MAIN CONTROL BOX ASSEMBLY-----	下控制總成(T把)含固定片配件--	18-1
INDEX 19	BATTERY CABLES-----	加電瓶模組(A)-----	19-1
INDEX 20	CONTROL AND BATTERY WIRING ILLUSTRATION-	控制及電瓶導線總成-----	20-1
	APPENDIX(1)-----	附錄(1)-----	

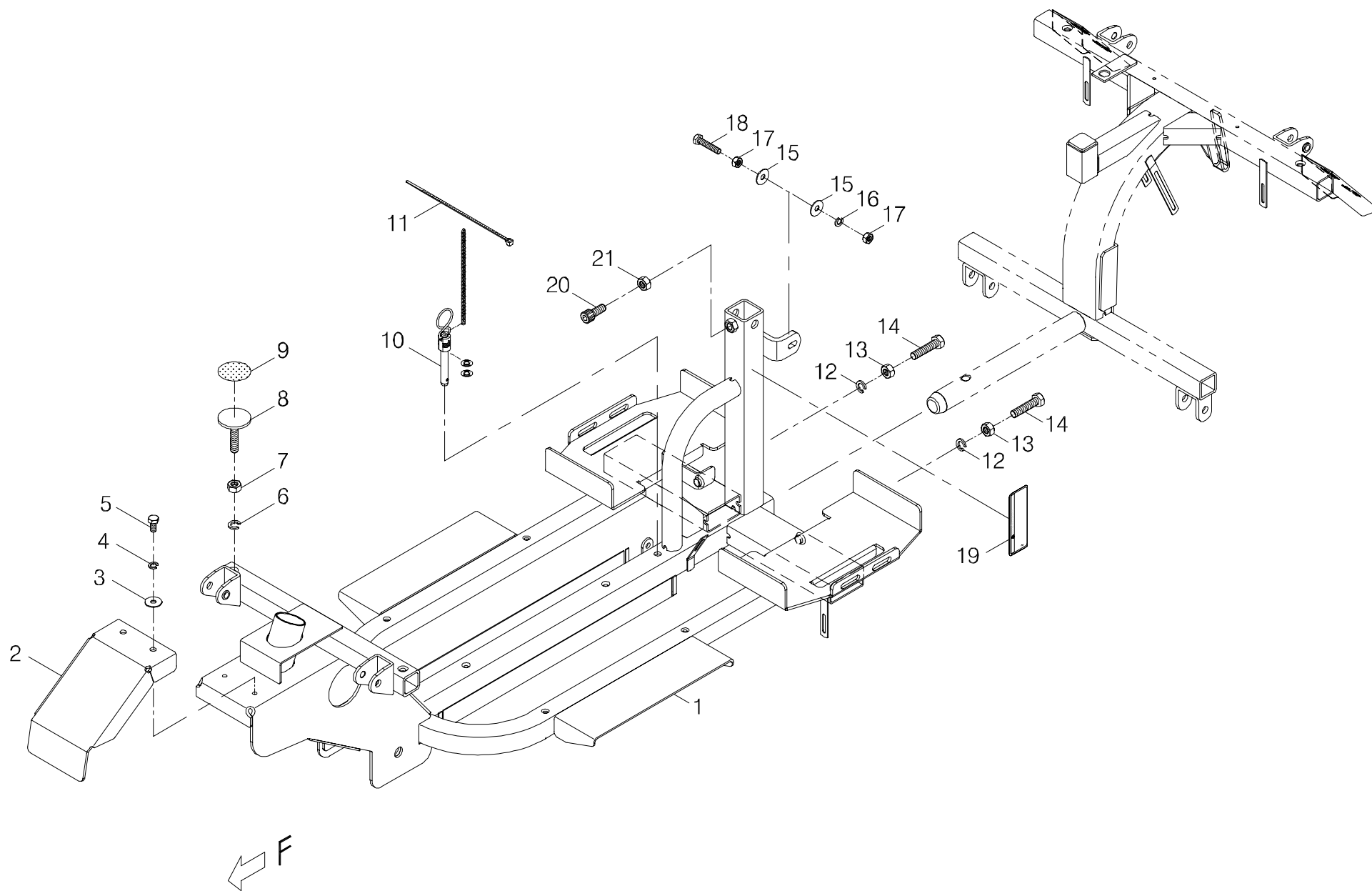


INDEX.1

Index NO.	Part NO.	Qty.	Description	Specification	Remark
1	-----		Front chassis assembly		前車架總成
1-1	102101-67101	1	Front chassis assembly	888SL/889SL	前車架總成/ 888SL/889SL
			-----*		
1-2	4001514-00	1	Bracket	A514	蓋板支撐架/A514(889SEL/SL)
1-3	5021-06018-210	2	Flat Washer	φ 6*18*1t,Multicolored Galvanized	平墊圈/ φ 6*18*1t (五彩)
1-4	5022-06010-210	2	Spring washer	φ 6*10,Multicolored Galvanized	彈簧墊圈/ φ 6*10 (五彩)
1-5	5001-06012-210	2	Hex head screw	M6*P1.0*12L,Multicolored Galvanized	外六角螺栓/M6*P1.0*12L (五彩)
			-----*		
1-6	5022-08013-210	2	Spring washer	φ 8*13,Multicolored Galvanized	彈簧墊圈/ φ 8*13 (五彩)
1-7	5011-08013-200	2	Hex Nut	M8*P1.25*13,Multicolored Galvanized	六角螺帽/M8*P1.25*13邊 (五彩)
1-8	2329-00200-01	2	SET , screw	M8*P1.25*35L,Multicolor	定位螺栓/M8*P1.25*35L (五彩)
1-9	2399-00300-00	2	Velcro	φ 35,Hook	自黏扣帶(勾面)/ φ 35
			-----*		
1-10	202922-00103	1	Safety Pin	φ 7.85,With two loops	安全銷(含鍊條12L)/ φ 7.85(二環)
1-11	505201-01500-50	1	Cable tie	CV-150B	束帶/CV-150B
			-----*		
1-12	5022-08013-210	2	Spring washer	φ 8*13,Multicolored Galvanized	彈簧墊圈/ φ 8*13 (五彩)
1-13	5011-08013-200	2	Hex Nut	M8*P1.25*13,Multicolored Galvanized	六角螺帽/M8*P1.25*13邊 (五彩)
1-14	5001-08035-210	2	Hex head screw	M8*P1.25*35L,Multicolored Galvanized	外六角螺栓/M8*P1.25*35L (五彩)
			-----*		
1-15	5021-06018-210	4	Flat Washer	φ 6*18*1t,Multicolored Galvanized	平墊圈/ φ 6*18*1t (五彩)
1-16	5022-06010-210	2	Spring washer	φ 6*10,Multicolored Galvanized	彈簧墊圈/ φ 6*10 (五彩)
1-17	5011-06010-200	4	Hex Nut	M6*P1.0*10*5L,Multicolored Galvanized	六角螺帽/M6*P1.0*10邊*5L (五彩)
1-18	5001-06030-210	2	Hex head screw	M6*P1.0*30L,Multicolored Galvanized	外六角螺栓/M6*P1.0*30L (五彩)

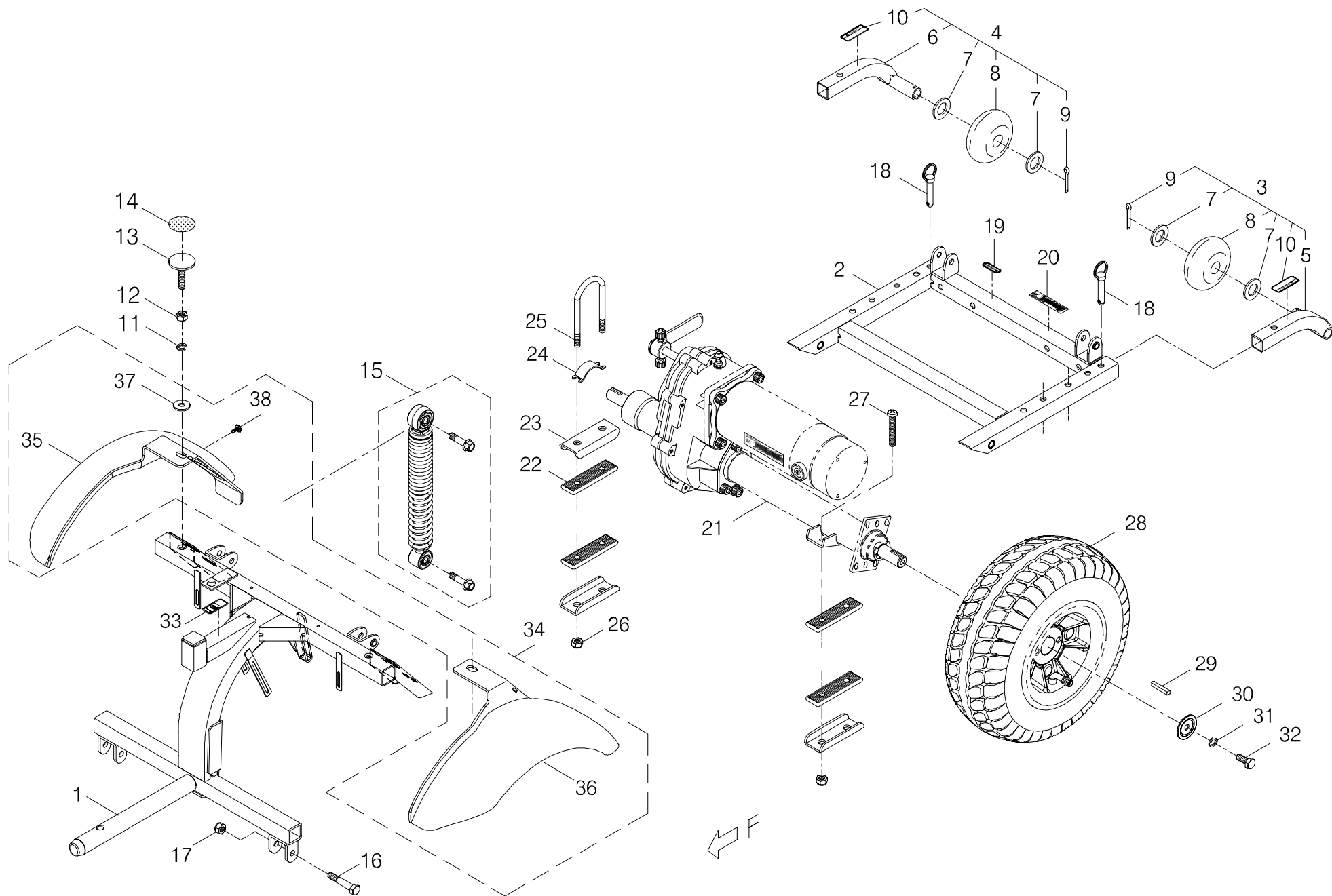



Index NO.	Part NO.	Qty.	Description	Specification	Remark
2	-----		Rear chassis assembly		後車架總成
2-1	102202-67101	1	Rear chassis assembly	Upper,888SL/889SL	後車架(上)總成/ 888SL/889SL
2-2	102203-67100-A2	1	Rear Chassis	Lower, 888SEL/889SEL, A2	後車架(下)總成/888SEL/889SEL(A2)
			-----*-----		
2-3	-----		Anti-tip wheel frame assembly		防傾輪架模組(左)
2-5	102918-89800C	1	Anti-tip wheel frame assembly	Left	防傾輪架(左)總成/889A
2-7	4001305-00	2	Flat Washer	A305, ϕ 17.5*30*2.6t	平墊圈/A305 (ϕ 17.5*30*2.6t)
2-8	2399-00100-00	1	anti-tip wheel	ϕ 17*75*30t	防傾輪/ ϕ 17*75*30t
2-9	5023-03025-210	1	Split pin	M3*25L,Multicolored Galvanized	開口銷/M3*25L (五彩)
2-10	301174-06	1	Warning sticker	N74-6	警告貼紙/N74-6
			-----*-----		
2-4	-----		Anti-tip wheel frame assembly	Right	防傾輪架模組(右)
2-6	102919-89800C	1	Anti-tip wheel frame assembly	Right,889A	防傾輪架(右)總成/889A
2-7	4001305-00	2	Washer	A305, ϕ 17.5*30*2.6t	平墊圈/A305 (ϕ 17.5*30*2.6t)
2-8	2399-00100-00	1	anti-tip wheel	ϕ 17*75*30t	防傾輪/ ϕ 17*75*30t
2-9	5023-03025-210	1	Split pin	M3*25L,Multicolored Galvanized	開口銷/M3*25L (五彩)
2-10	301174-06	1	Warning sticker	N74-6	警告貼紙/N74-6
			-----*-----		
2-11	5022-08013-210	2	Spring washer	ϕ 8*13,Multicolored Galvanized	彈簧墊圈/ ϕ 8*13 (五彩)
2-12	5011-08013-200	2	Hex Nut	M8*P1.25*13,Multicolored Galvanized	六角螺帽/M8*P1.25*13邊 (五彩)
2-13	2329-00200-01C	2	SET , screw	M8*P1.25*35L,Multicolor	定位螺栓/M8*P1.25*35L (五彩)
2-14	2399-00300-00	2	Velcro	ϕ 35,Hook	自黏扣帶(勾面)/ ϕ 35
			-----*-----		
2-15	202931-67100-A2	2	Rear Suspension w/ Bolts	888SEL/889SEL(A2)	後避震器/888SEL/889SEL(含螺栓)(A2)


Index NO.	Part NO.	Qty.	Description	Specification	Remark
2-16	5001-08045-220	2	Hex Head Bolt	M8*P1.25*45L,Multicolored Galvanized	外六角鋼栓/M8*P1.25*45L 半牙 (五彩)
2-17	5012-08013-200	2	Nyloc Nut	M8*P1.25*13 7.8t,Multicolored Galvanized	尼龍螺帽/M8*P1.25*13邊7.8t(五彩)
2-18	202922-88100	2	Pin	M7.8*40L,PP08040B	連接銷/M7.8*40L (PP08040B)
2-19	301174-09	1	Warning sticker	N74-9	警告專利貼紙/N74-9
2-20	-----	REF(1)	Decal		條碼貼紙
			-----*-----		
2-21	-----	REF(1)	Drive-train assembly		馬達剎車變速箱總成
			-----*-----		
2-22	209909-88100	4	Rubber Gasket	85*25*7t	減震膠片(二孔距47mm)/85*25*7t
2-23	4001168-00	3	Bracket	A168	固定片/A168
2-24	4001192-00	1	Arc Plate	A192	弧形片/A192
2-25	4003210-01	1	U-Type Screw	C210-1	U型螺栓/ C210-1
2-26	5012-08013-200	4	Nyloc Nut	M8*P1.25*13 7.8t,Multicolored Galvanized	尼龍螺帽/M8*P1.25*13邊7.8t(五彩)
2-27	5005-08050-210	2	Philip Screw	M8*P1.25*50L,Multicolored Galvanized	十字圓頭螺栓/M8*P1.25*50L (五彩)
			-----*-----		
2-28	-----	REF(2)	Rear wheel assembly		後輪總成
2-29	505301-00505-02	2	Drive Key	5*5*35	單頭圓鍵/5*5*35
2-30	4001329-00	2	Flat Washer	A329	平墊圈/A329
2-31	5022-08013-210	2	Spring washer	φ 8*13,Multicolored Galvanized	彈簧墊圈/ φ 8*13 (五彩)
2-32	5001-08020-440	2	Hex Head Bolt	M8*P1.25*20L,SUS304	外六角鋼栓(8.8級)/M8*P1.25*20L SUS304
			-----*-----		
2-33	301174-07-A2	1	Warning Sticker	N74-7-A2	警告貼紙/N74-7-A2
			-----*-----		
2-34	800907-675-02	1	Fender Kit w/ Hardware	for 889SL w/ 4.00-5 Tire	擋泥板(含螺栓)模組/889SL(配合寬胎)

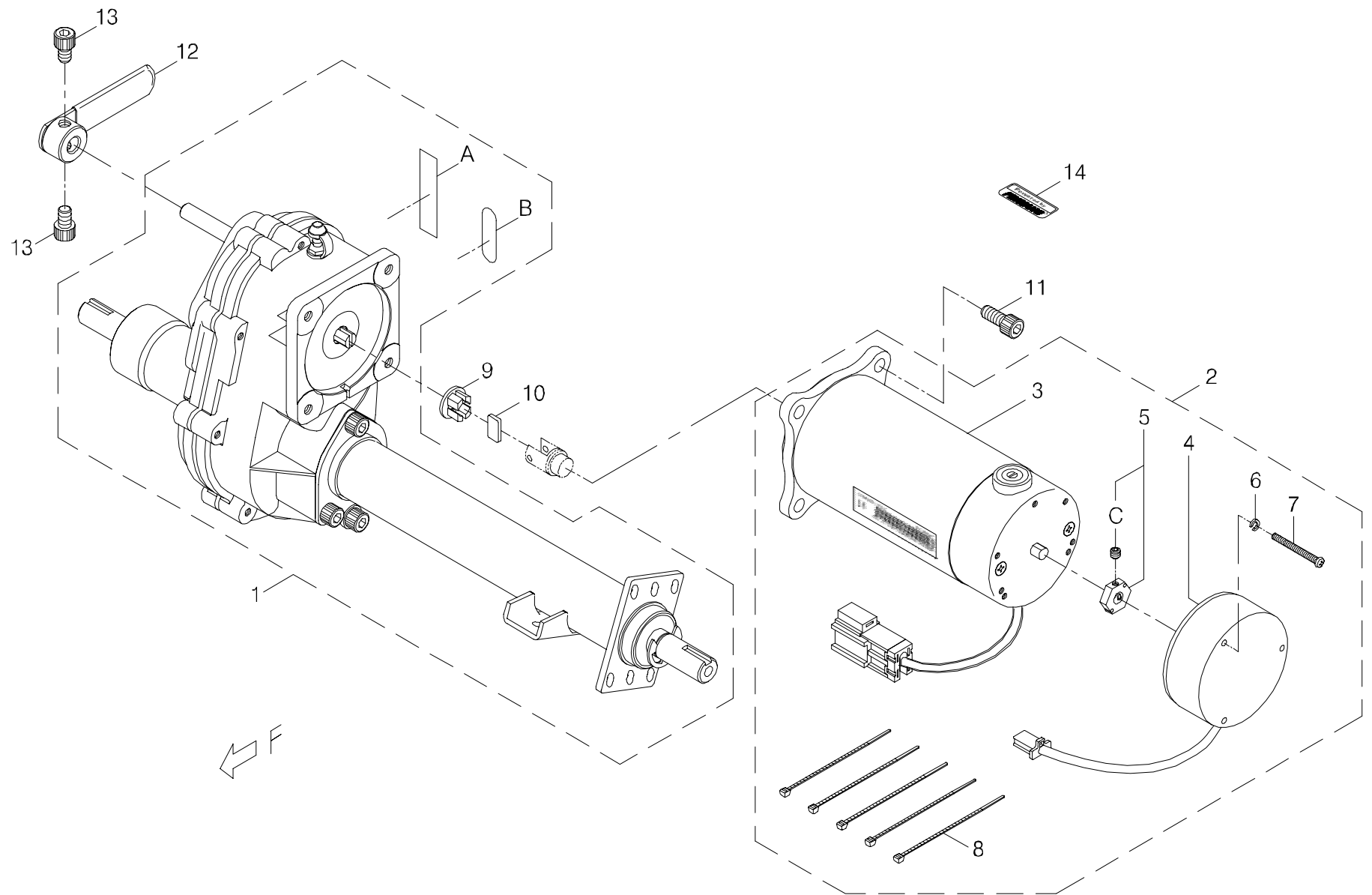

Index NO.	Part NO.	Qty.	Description	Specification	Remark
3	100001-67501-A5	1	Drive-train Assembly	9MNF-2B, #6, 889SEL NTBG	馬達剎車變速箱總成/9MNF-2B+(#6)+889SEL NTBG
3-1	100401-88900-A3	1	Transaxle assembly	889SEL NTBG(#1-1),Black(A3)	變速箱總成/889SEL NTBG(#1-1)黑色(A3)
3-A	-----	REF(1)	Decal		條碼貼紙
3-B	301174-09	REF(1)	Patent warning sticker		警告專利貼紙/N74-9
3-2	100101-89506-A5	1	Motor and Brake Assembly	9MNF-2B, #6	馬達剎車總成/9MNF-2B+#6
3-3	100201-89608	1	Motor Assembly	9MNF-2B	馬達(總成)/9MNF-2B
3-4	100309-88100	1	Electromagnetic brake assembly	#6	電磁剎車總成#6/(含剎車線)
3-5	200301-66400	1	Driving Hex Axle for Brake Disc	#1, ϕ 8*19.1*6L	剎車六角軸套/ #1 (ϕ 8*對邊19.1*6L)
3-C	-----	REF(2)	Nylock socket screw		耐落無頭螺栓/M4*P0.7*5L(附屬品)
3-6	5022-04007-210	3	Spring washer	ϕ 4*7*1.25t,Multicolored Galvanized	彈簧墊圈/ ϕ 4*7*1.25t (五彩)
3-7	5005-04040-210	3	Screw	M4*P0.7*40L,Multicolored Galvanized	十字圓頭螺栓/M4*P0.7*40L (五彩) 全牙
3-8	505201-00940-50	2	cable Tie	CV-094B	束帶/CV-094B
3-9	200904-88100	1	Joint Adapter		變速箱連接襯套
3-10	200905-88100	1	Plate for Joint Adapter		變速箱連接片
3-11	5002-08020-220	4	CAP HEAD ALLEN SCREW	M8*P1.25*20L,Multicolored Galvanized	圓頭內六角鋼栓 12.9級/M8*P1.25*20L (五彩)
3-12	200902-88100	1	Freewheel Lever	100L	離合器拉柄/ 100L
3-13	5002-08012-220	2	CAP HEAD ALLEN SCREW	M8*P1.25*12L,Multicolored Galvanized	圓頭內六角鋼栓/M8*P1.25*12L (五彩)
3-14	301179-04	1	Trademark Sticker	N79-4	指示貼紙(40*15.5)/ N79-4(Powered by PIHSIANG)
-----	9005-004	REF	Transmission grease	GP-000/180Kg	變速箱潤滑油/GP-000/180Kg

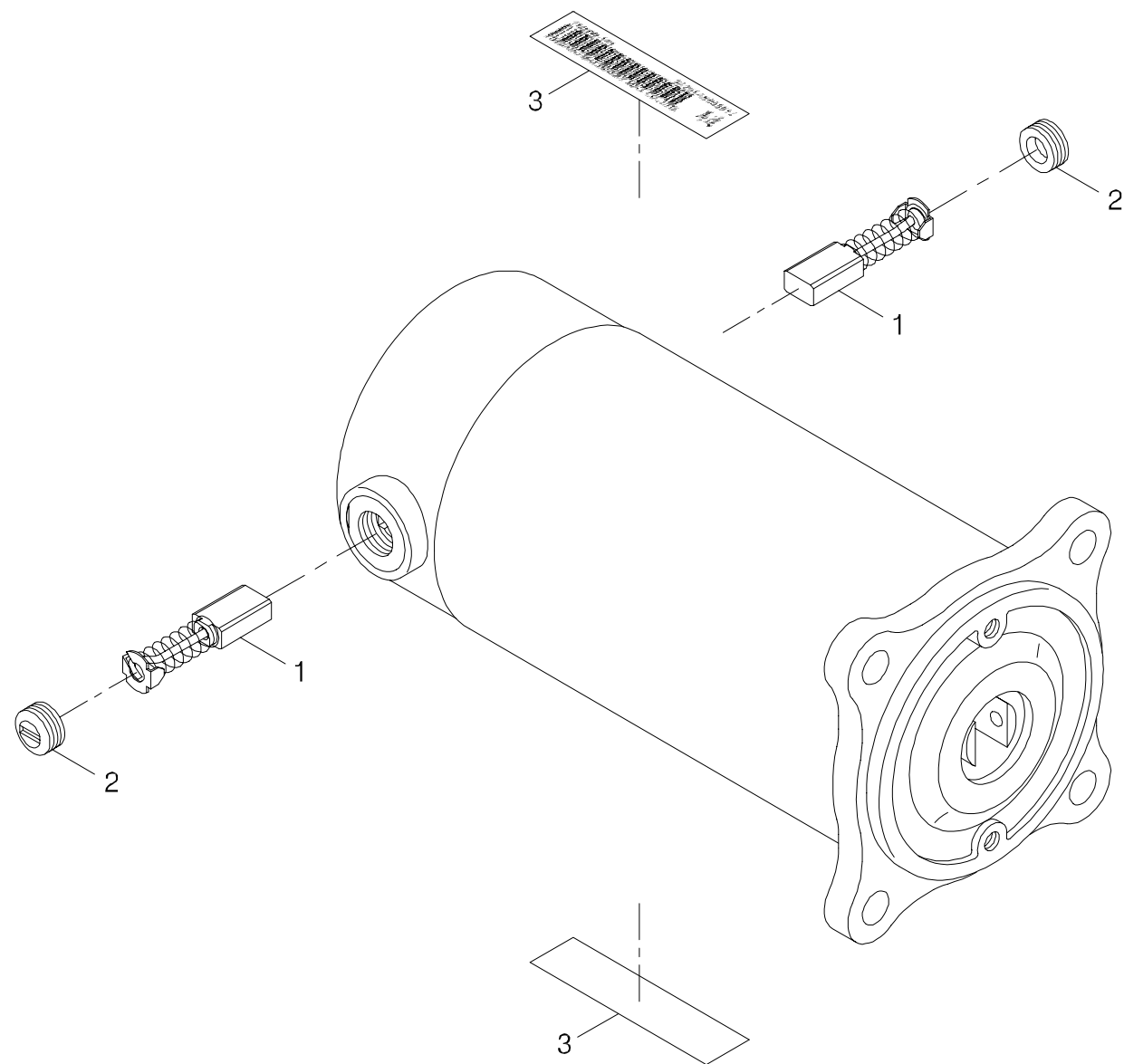


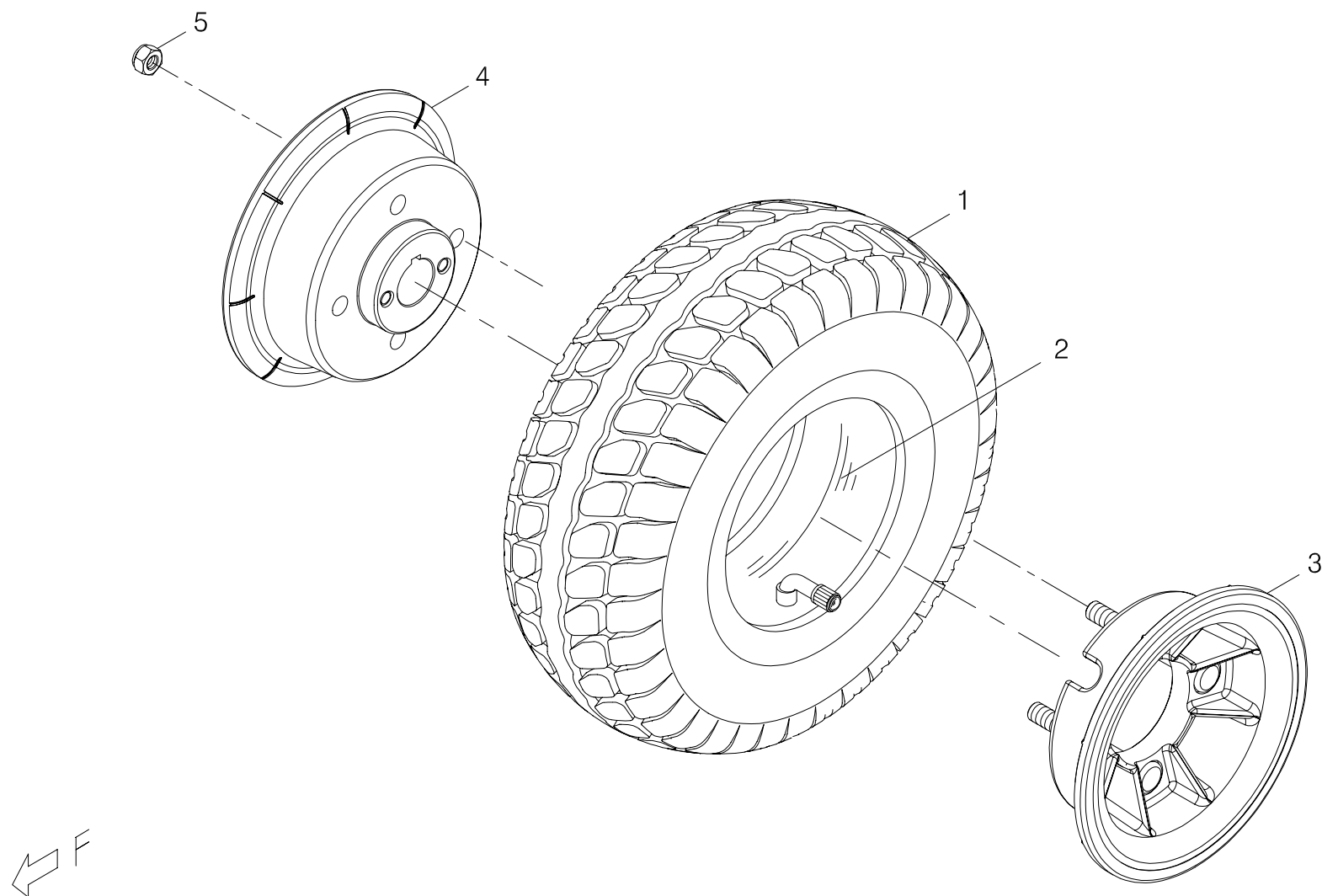


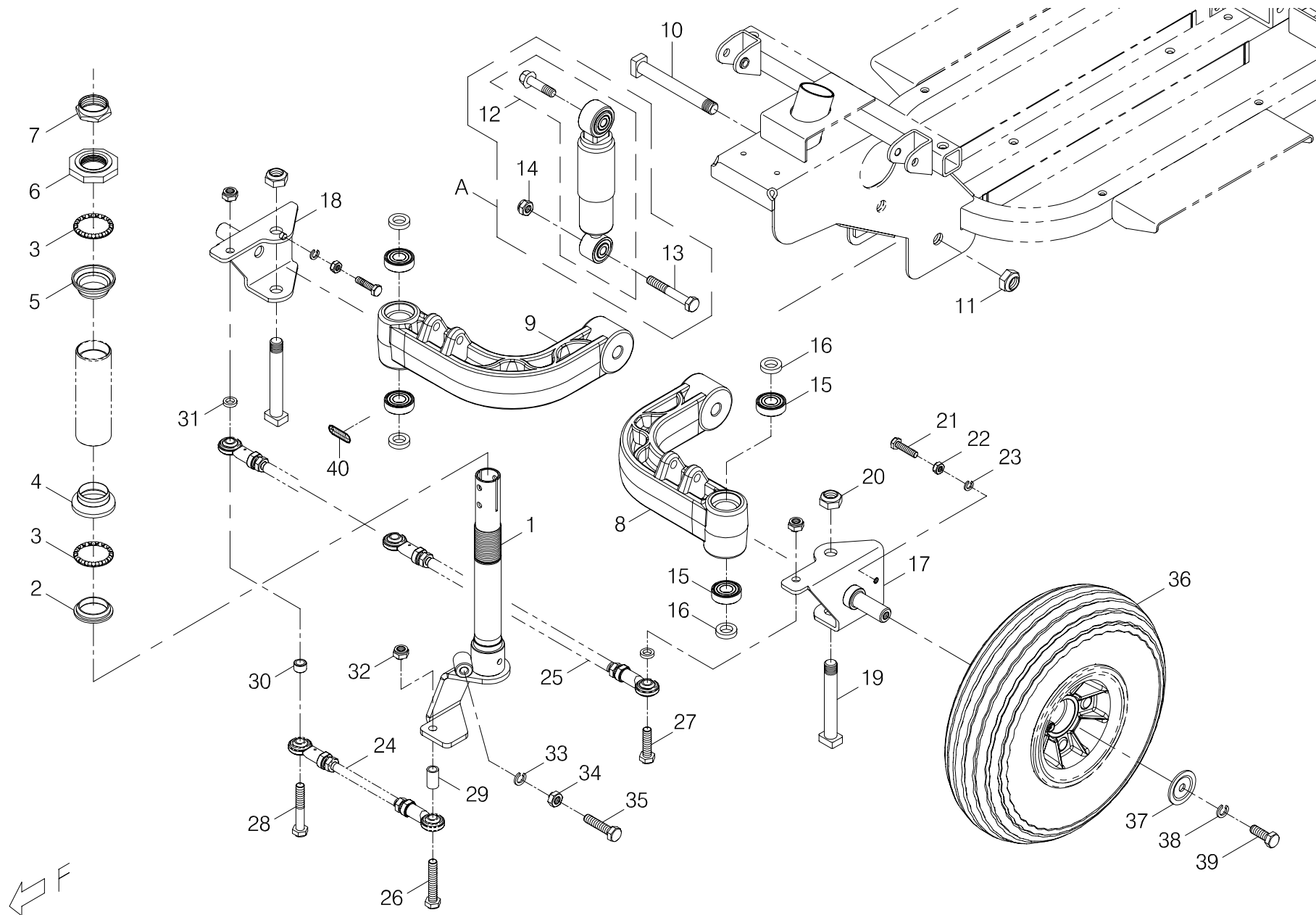



Index NO.	Part NO.	Qty.	Description	Specification	Remark
6	-----		Front axle assembly		導向輪總成
6-1	101501-67100C	1	Steering Shaft	888SEL, 889SEL	操向桿/888SEL/889SEL
			-----*-----		
6-2	201401-88200	1	Bearing bushing	φ 27	軸承座/ φ 27
6-3	201402-88200	2	Steering Bearings	5/32*16	軸承/5/32"*16
6-4	201403-88200	1	Bearing housing	φ 30	珠碗(下)/ φ 30
6-5	201404-88200	1	Bearing housing	φ 30	珠碗(上)/ φ 30
6-6	201405-88200	1	Bearing bushing	Large octangle	軸承襯套/大八角
6-7	201406-88200	1	Steering Head Nut	Dia 22.2	軸承固定螺帽/ φ 22.2
			-----*-----		
6-8	101802-67100-A2	1	Front Crossbeam	Left,888SEL,889SEL#1, A2	前輪橫樑(左)/888SEL/889SEL#1
6-9	101803-67100-A2	1	Front Crossbeam	Right,888SEL,889SEL#1, A2	前輪橫樑(右)/888SEL/889SEL#1
6-10	4003048-07	1	Front fork shaft	C48-7,M12*P1.25*98L,Multicolor	前叉軸/C48-7 (M12*P1.25*98L)五彩
6-11	5014-12019-210	1	Self-lock nut	M12*P1.25*19,Multicolored Galvanized	防鬆螺帽/M12*P1.25*19邊 (五彩)
			-----*-----		
6-A	102930-89501-A3	2	Front Suspension Assembly	889DXN/SEL, 888SEL, A3	前避震器總成/889DXN/SEL,888SEL(A3)(保稅品)
6-12	202930-89500-A2C	1	Front Suspension	889DXN, A2, w/ Screw	前避震器/889DXN(含螺栓)(A2)
6-13	5001-08050-221	1	Hex Head Bolt	M8*P1.25*50L, Multicolored Galvanized	外六角鋼栓(8.8級)/M8*P1.25*50L牙長12L(五彩)
6-14	5012-08013-200	1	Nyloc Nut	M8*P1.25*13 7.8t,Multicolored Galvanized	尼龍螺帽/M8*P1.25*13邊7.8t(五彩)
			-----*-----		
6-15	505101-60012-01	4	Ball bearing	#6001zz,NBN	滾珠軸承/#6001zz (NBN)
6-16	4005006-00	4	Spacer	H6, φ 12ID* φ 20OD*4.5T	間隔圈/H6(φ 12 ID* φ 20 OD*4.5T)
			-----*-----		
6-17	202902-66801-A2C	1	Front Wheel Axle	Left, J-8, #1, A2	前輪軸(左)/J-8 #1(A2)

Index NO.	Part NO.	Qty.	Description	Specification	Remark
6-18	202903-66801-A2C	1	Front Wheel Axle	Right, J-8, #1, A2	前輪軸(右)/J-8 #1(A2)
6-19	4003048-06	2	Front fork shaft	C48-6,M12*P1.25*85L,Multicolor	前叉軸/C48-6 (M12*P1.25*85L)五彩
6-20	5014-12019-210	2	Self-lock nut	M12*P1.25*19,Multicolored Galvanized	防鬆螺帽/M12*P1.25*19邊 (五彩)
6-21	5001-06025-220	2	Hex Head Bolt	M6*P1.0*25L,Multicolored Galvanized	外六角鋼栓/M6*P1.0*25L (五彩)8.8級
6-22	5011-06010-200	2	Hex Nut	M6*P1.0*10*5L,Multicolored Galvanized	六角螺帽/M6*P1.0*10邊*5L (五彩)
6-23	5022-06010-210	2	Spring Washer	φ 6*10,Multicolored Galvanized	彈簧墊圈/ φ 6*10 (五彩)
			-----*-----		
6-24	-----	REF(1)	Connecting rod assembly (Short)		導向連桿總成(短)
6-25	-----	REF(1)	Connecting rod assembly (Long)		導向連桿總成(長)
6-26	5001-08045-211	1	Hex head screw	M8*P1.25*45L,Multicolored Galvanized	外六角螺絲/M8*P1.25*45L 全牙 (五彩)
6-27	5001-08030-210	1	Hex head screw	M8*P1.25*30L,Multicolored Galvanized	外六角螺絲/M8*P1.25*30L (五彩)
6-28	5001-08050-220	1	Hex Head Bolt	M8*P1.25*50L,Multicolored Galvanized	外六角鋼栓/M8*P1.25*50L 半牙(五彩)
6-29	4003264-11	1	Spacer	C264-11, φ 8*12*19L	間隔圈/C264-11 (φ 8*12*19L)
6-30	4003264-05	1	Spacer	C264-5, φ 8.1* φ 12*8L	間隔圈/C264-5 (φ 8.1* φ 12*8L)
6-31	4001068-01	2	Flat Washer	A68-1, φ 8.1*13*3t	平墊圈/A68-1 (φ 8.1*13*3t) 銅
6-32	5012-08013-200	3	Nyloc Nut	M8*P1.25*13 7.8t,Multicolored Galvanized	尼龍螺帽/M8*P1.25*13邊7.8t(五彩)
			-----*-----		
6-33	5022-08013-210	1	Spring washer	φ 8*13,Multicolored Galvanized	彈簧墊圈/ φ 8*13 (五彩)
6-34	5011-08013-200	1	Hex Nut	M8*P1.25*13,Multicolored Galvanized	六角螺帽/M8*P1.25*13邊 (五彩)
6-35	5001-08035-210	1	Hex head screw	M8*P1.25*35L,Multicolored Galvanized	外六角螺絲/M8*P1.25*35L (五彩)
			-----*-----		
6-36	-----	REF(2)	Front wheel assembly		前輪總成
6-37	4001329-00	2	Flat Washer	A329	平墊圈/A329
6-38	5022-08013-210	2	Spring washer	φ 8*13,Multicolored Galvanized	彈簧墊圈/ φ 8*13 (五彩)

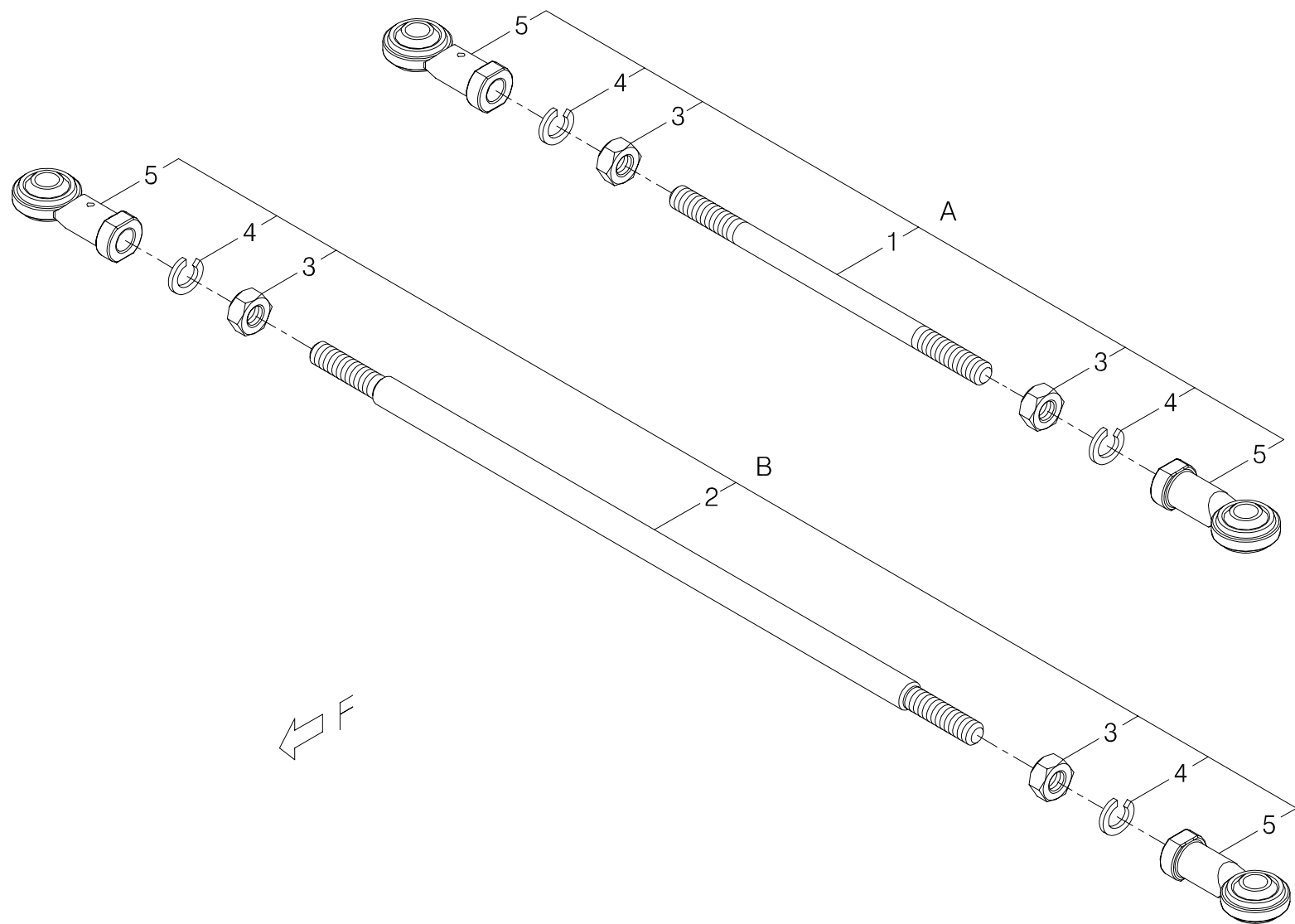


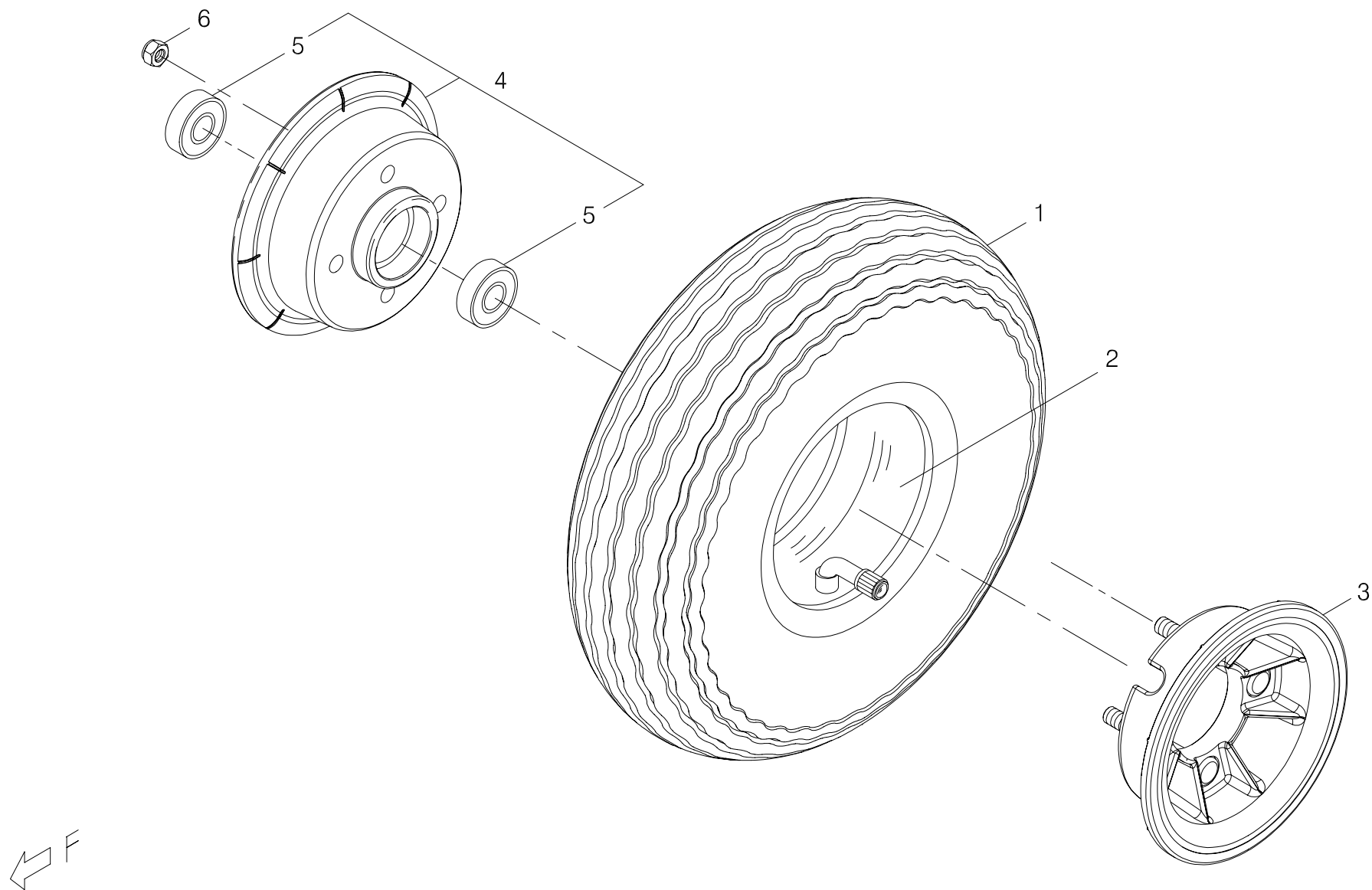


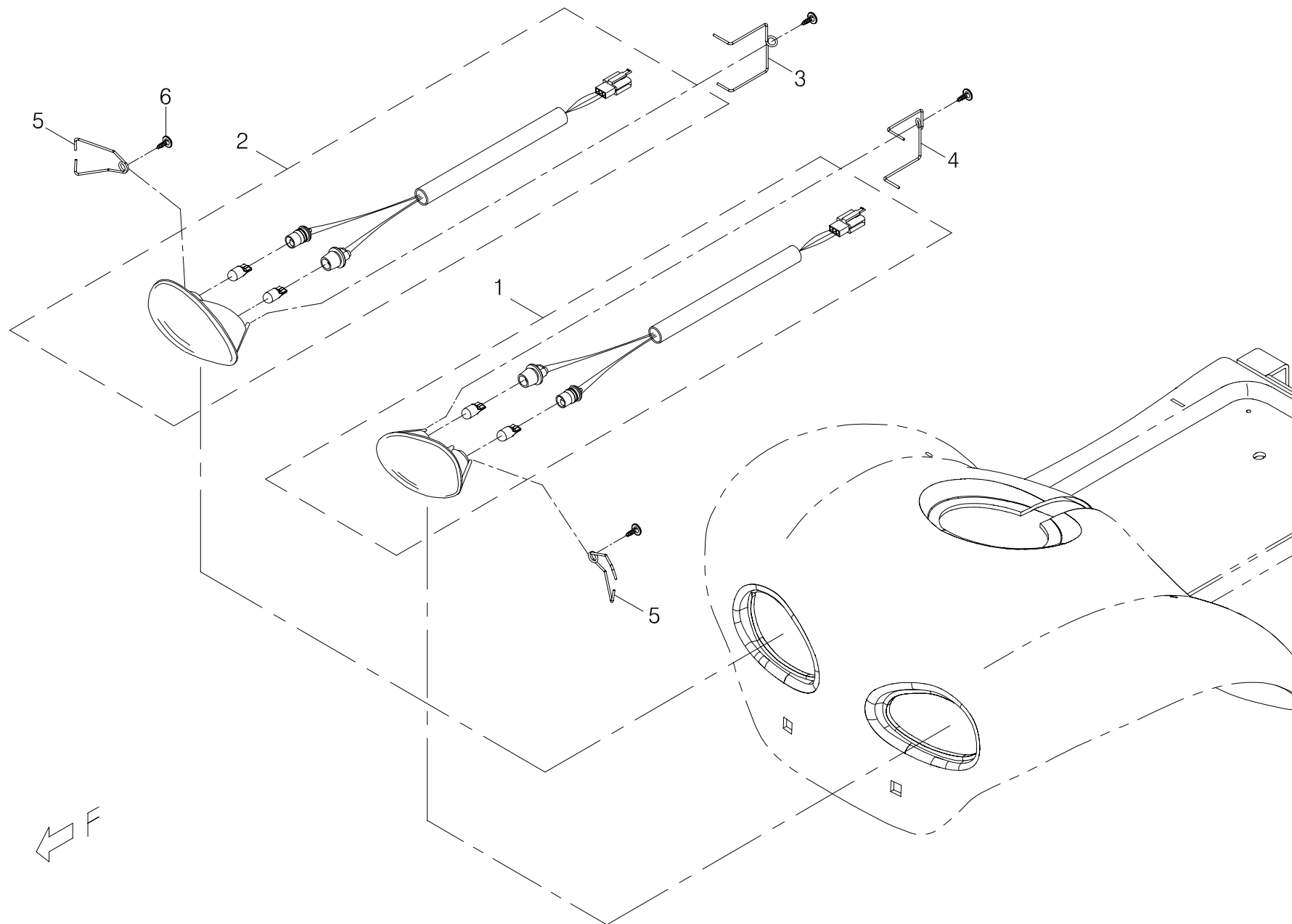

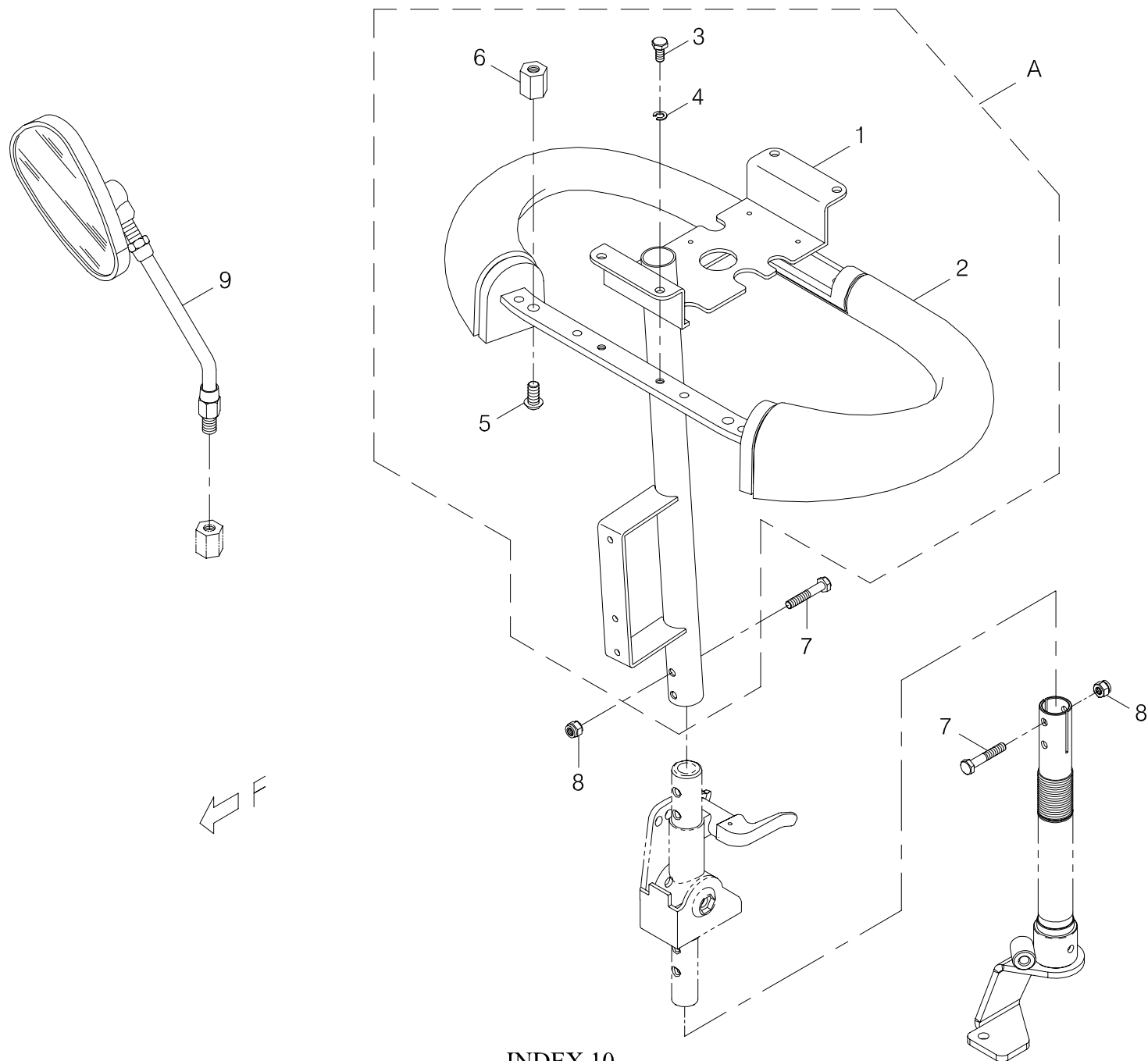


INDEX.10

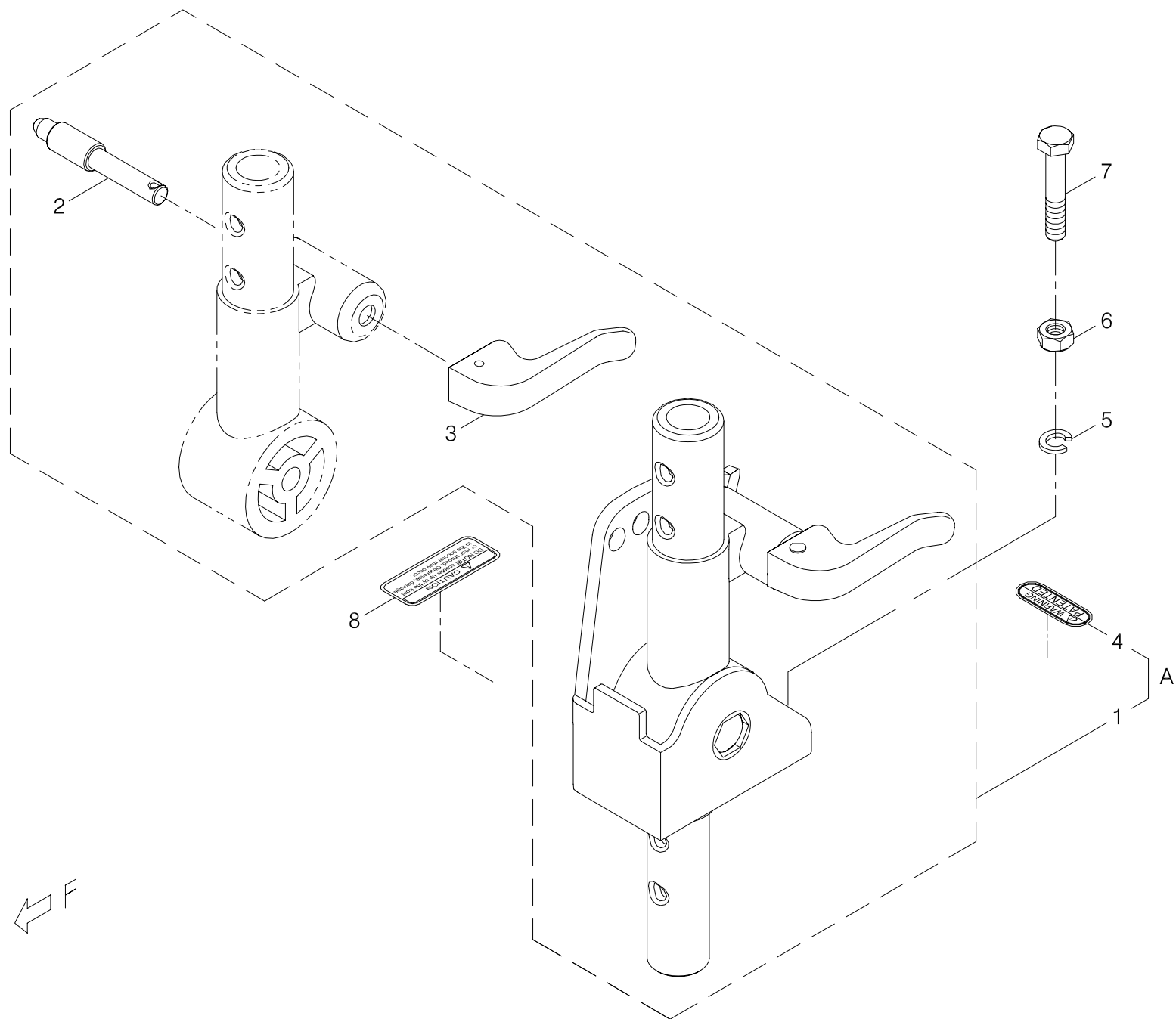


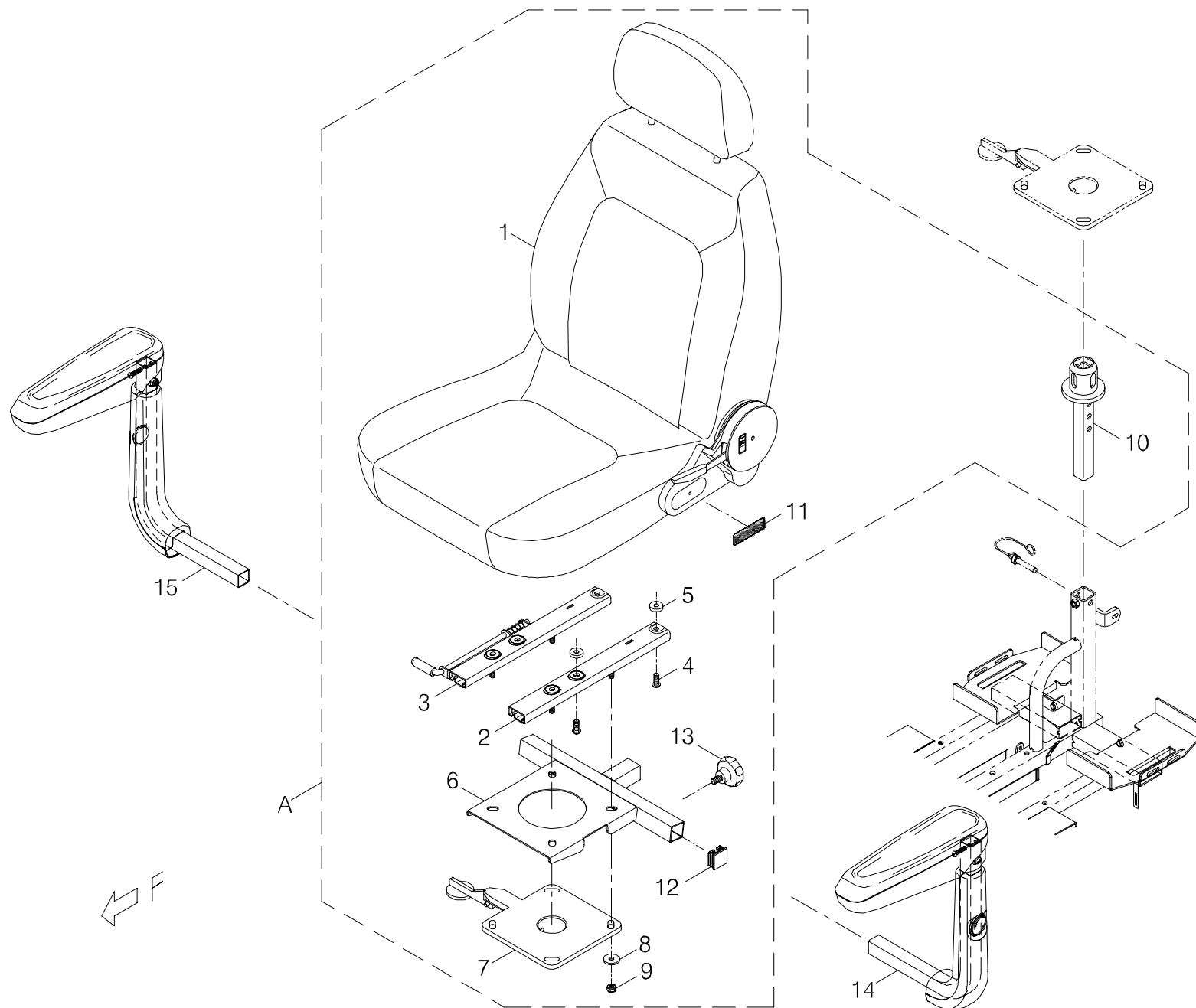


Index NO.	Part NO.	Qty.	Description	Specification	Remark
12	-----		Swivel seat, adaptor assembly		旋轉-座台，支撐座總成
12-A	800317-675-00	1	Deluxe Seat Assembly w/ Slider	Black Vinyl, Reclining, 889SLBF	高背大椅模組+滑道/黑皮+後傾器(全包覆式)889SLBF
12-1	104021-00108	1	Black Vinyl Captain Seat,High Backrest	18.5"x17.5",w/Reclining 112,Headrest	高背皮椅6段112度後傾DL頭枕/黑色(包覆)(18.5*17.5)(薄座)
12-2	204901-88803-A2	1	Sliding Rail	Left, #5, A2	滑道(左)/#5(A2)
12-3	204902-88803-A2	1	Sliding Rail	Right, #5, A2	滑道(右)/#5(A2)
12-4	5003-08020-221	4	Allen Button Head Screw	M8*P1.25*20L,Multicolored Galvanized	半圓頭內六角鋼栓10.9級/M8*P1.25*20L(防鬆全支)五彩
12-5	5021-08022-211	4	Flat Washer	Dia 8*22*5t,Multicolored Galvanized	平墊圈/φ 8*22*5t (五彩)
12-6	204506-66101	1	Seat Adapter	#2	座台/ #2
12-7	204507-66103C	1	Swivel Seat Stand	#4, Aluminum	旋轉台(鋁)#4
12-8	4001428-00	4	Flat Washer	A428, φ 8.4*26*3t	平墊圈/A428 (φ 8.4*26*3t)
12-9	5012-08013-200	4	Nyloc Nut	M8*P1.25*13 7.8t,Multicolored Galvanized	尼龍螺帽/M8*P1.25*13邊7.8t(五彩)
12-10	204508-88805-A3C	1	Seat Post Adaptor	#2, 165L, w/ 3 Holes, A3	旋轉台基座#2/165L(3 孔)A3
12-11	301174-31	1	Caution Sticker	N74-31	警告貼紙/N74-31
12-12	209931-00105	2	Square Plug	30MM*30MM,2T	方形管塞/30MM*30MM 2T管用
12-13	5008-10020-212-A2	2	Tighten Knob	M10*17L,Multicolored Galvanized,A2	星型螺栓/M10*17L全牙,五彩 (A2)
			-----*		
12-14	104300-88800	1	Armrest Assembly	A1,Left	扶手總成(左)/A1
12-15	104400-88800	1	Armrest Assembly	A1,Right	扶手總成(右)/A1

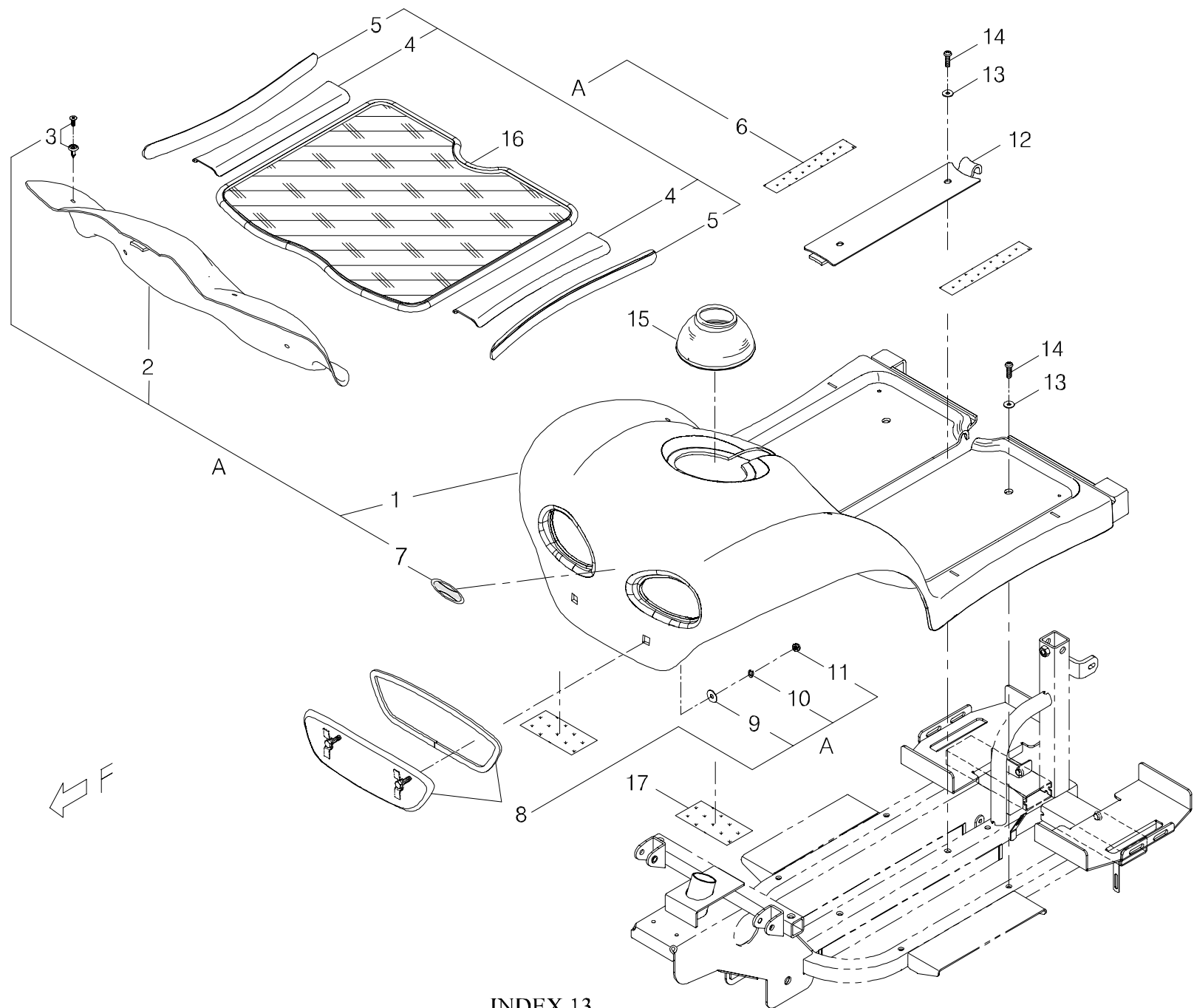

Index NO.	Part NO.	Qty.	Description	Specification	Remark
13-A	103102-67500-22	1	Front chassis cover assembly	w/decal,889SEL,Burgundy	前蓋板總成(含貼紙)/889SEL 棗紅
13-1	103101-67500-22	1	Front chassis cover assembly	889SEL,Burgundy	前蓋板總成/889SEL 棗紅
13-2	203505-67500	1	Protection cover	889SEL	腳踏護蓋/889SEL
13-3	209912-78500-A3	4	Plastic Stud	G3020060	腳釘/G3020060
13-4	203501-67100	2	Protect cover	888SEL/889SEL	前蓋板護片/888SEL/889SEL
13-5	203102-88800	2	Striped tape	Front,24.5W*330L	飾條(前)/24.5W*330L
13-6	203103-66500	2	Velcro	25W*25M,Hook	自黏扣帶(勾面)/25W*25M
13-7	301217-13-A2	1	Trademark Decal, SHOPRIDER	T17-13-A2, 63.7*26	商標貼紙/T17-13-A2 (SHOPRIDER) 63.7*26
13-8	202901-67101C	1	Front Bumper	888SEL/889SEL, #1	前保險桿(鎖附蓋板)/888SEL/889SEL#1(鍍鉻)
13-9	5021-06018-210	2	Flat Washer	φ 6*18*1t,Multicolored Galvanized	平墊圈/ φ 6*18*1t (五彩)
13-10	5022-06010-210	2	Spring Washer	φ 6*10, Multicolored Galvanized	彈簧墊圈/ φ 6*10 (五彩)
13-11	5012-06010-200	2	Nylon Lock Nut	M6*P1.0*10*6t,Multicolored Galvanized	尼龍螺帽/M6*P1.0*10邊*6t (五彩)
			-----*-----		
13-12	203510-67100-50	1	Groove Cover	888SEL/889SEL,Black,#1	溝槽護蓋/888SEL/889SEL(黑)#1
13-13	5021-06018-440	6	Flat Washer	φ 6*18*1.6t,SUS	平墊圈/ φ 6*18*1.6t SUS
13-14	5005-06020-440	6	Screw	M6*P1.0*20L,SUS	十字圓頭螺栓/M6*P1.0*20L SUS
			-----*-----		
13-15	209903-78100	1	Rubber Cover	A2	防塵套(前)/A2
13-16	209906-67500	1	Carpet	889SEL	地毯(加包邊)/889SEL
13-17	203103-66400	2	Velcro	50W*25M,Loop	自黏扣帶(毛面)/50W*25M

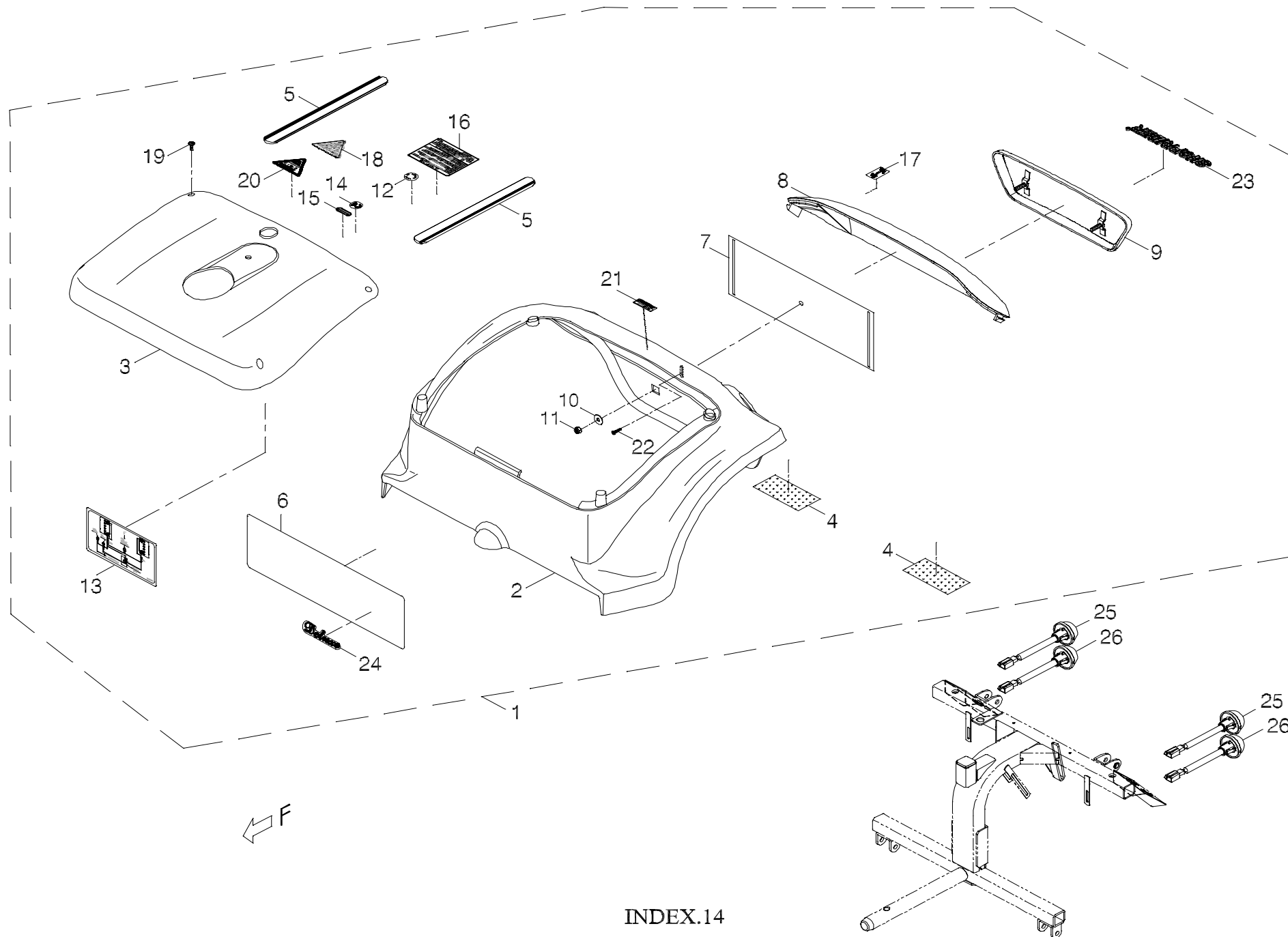


Index NO.	Part NO.	Qty.	Description	Specification	Remark
14-1	103202-67504-22	1	Rear chassis cover assembly	w/decal,889SEL,Burgundy	後蓋板總成(含貼紙)/889SEL 棗紅/(雙色燈罩)(N87-11)
14-2	103201-67500-22-A2	1	Rear chassis cover assembly	889SEL,Burgundy#1	後蓋板總成/889SEL 棗紅#1
14-3	103207-88800-14	1	Chassis cover assembly	888SEL/889SEL,Silver	後上蓋總成/888SEL/889SEL 鐵灰銀
14-4	203103-66400	2	Velcro	50W*25M,Loop	自黏扣帶(毛面)/50W*25M
14-5	203202-88800	2	Striped tape	Rear,24.5W*265L	飾條(後)/24.5W*265L
14-6	203203-88800	1	Emblem	Rear,110W*350L	木紋紙(後)/110W*350L
14-7	203206-67500-A2	1	Reflector	Ivory,888SEL/889SEL#1	雷射反光片/888SEL/889SEL#1
14-8	203512-88800-A2	1	Charger inlet cover	888SEL/889SEL,Two-tone	燈罩/888SEL/889SEL(雙色)
14-9	209905-67501-A2	1	Rear bumper	889SEL#1	後保險桿/889SEL#1(A2)
14-10	5021-06018-210	2	Flat Washer	φ 6*18*1t,Multicolored Galvanized	平墊圈/ φ 6*18*1t (五彩)
14-11	5012-06010-200	2	Nylon Lock Nut	M6*P1.0*10*6t,Multicolored Galvanized	尼龍螺帽/M6*P1.0*10邊*6t (五彩)
14-12	301116-01	1	MIT Sticker	N16-1	MIT貼紙/N16-1
14-13	301153-06-A2	1	Battery Wiring Diagram Sticker, A2	N53-6-A2, AC2-150, 50A Circuit Breaker	指示貼紙(線路圖)/N53-6-A2 (AC2-150)(50A)
14-14	301166-03	1	CE Mark Label	N66-3	CE貼紙/N66-3 透黑
14-15	301174-09	1	Warning Sticker	N74-9	警告專利貼紙/N74-9
14-16	301174-13	1	Warning sticker	N74-13,A0518,North America	警告貼紙/N74-13(A0518) 美規
14-17	301187-11	1	ON-BRAKE-OFF Label	N87-11	ON-BRAKE-OFF圖示貼紙/N87-11 (透白)(代步車用)
14-18	301123-23	1	Warning sticker	N23-23	三角型警告貼紙/N23-23
14-19	5006-04016-710	4	Self tapping screw	M4*16L,Black Galvanized	自攻螺栓(十字扁圓頭)/M4*16L(鍍黑鋅)
14-20	301123-07	1	Warning sticker	N23-7,Large	三角型警告貼紙/N23-7 (大)
14-21	301179-04	1	Trademark Sticker	N79-4	指示貼紙(40*15.5)/N79-4(Powered by PIHSIANG)
14-22	5006-04012-310	1	Self Tapping Screw	M4*12L,Black,888A	自攻螺栓(鑽尾)帶墊/M4*12L 黑 (888A)
14-23	209939-68500	1	Trademark	SHOPRIDER	浮凸自黏商標/SHOPRIDER(電鍍鉻)
14-24	301206-00	1	Decal	T6,Deluxe	豪華型貼紙/T6 (Deluxe)

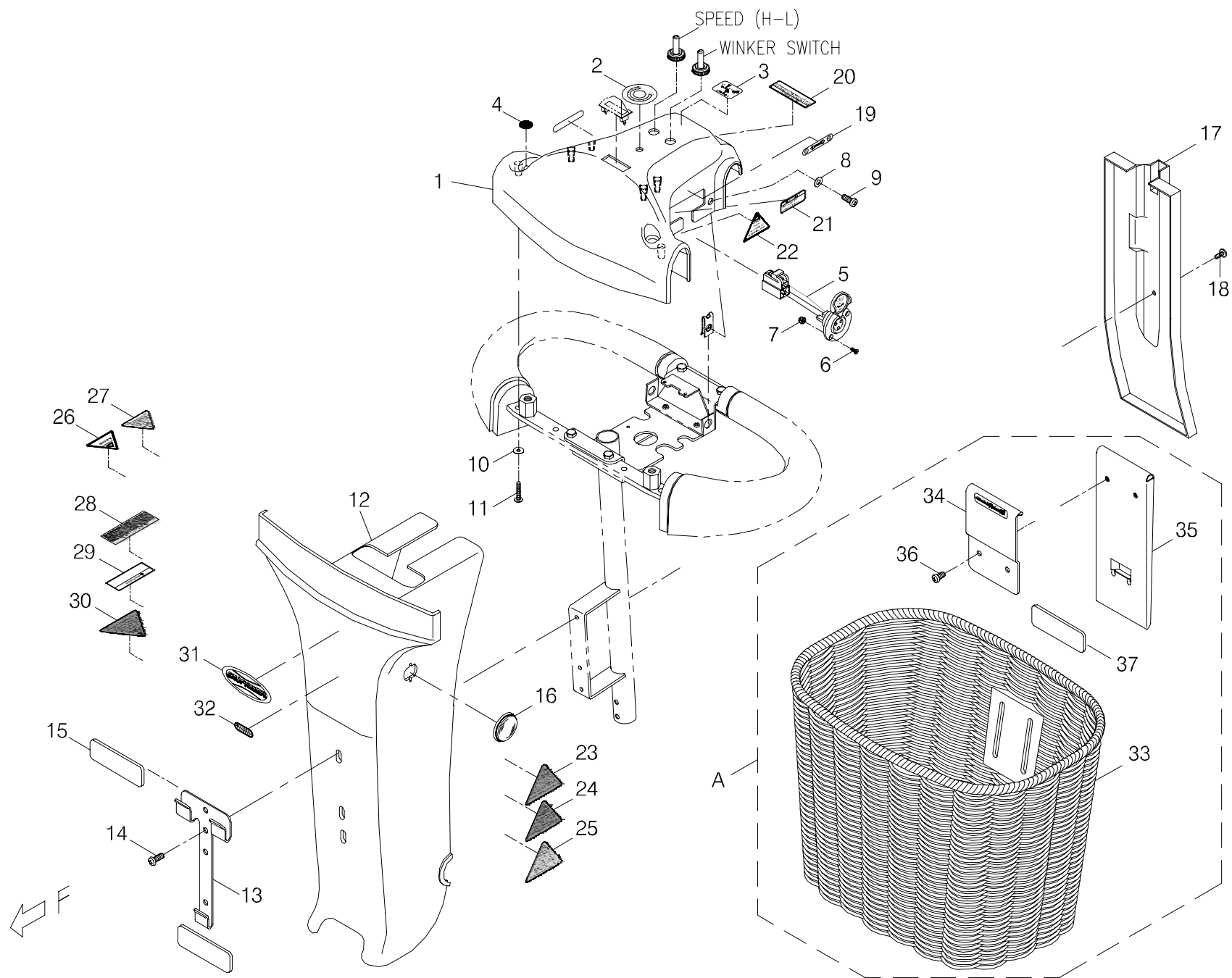


Index NO.	Part NO.	Qty.	Description	Specification	Remark
15	-----		Handle bar cover assembly		把手蓋總成
15-1	103303-66802-72	1	Front tiller cover	J-8,B1#1	環把上蓋(曲面披覆)/J-8 蛇紋色 B1(無孔)#1
15-2	301222-00-A2	1	Decal	T22	速率貼紙/T22 (10段)A2
15-3	301111-05	1	Instruction sticker	N11-5,HL,Turn Signal,ON/OFF	指示貼紙/ N11-5(高低速+方向燈+ON/OFF)
15-4	301114-07	2	Round sticker	N14-7,PVC	圓貼紙(黑)/N14-7(PVC)
15-5	207006-88109	1	Charging cable	KA-888-H11	充電線(短)/KA-888-H11
15-6	5009-03010-440	2	Taper Head Philip Screw	M3*P0.5*10L,SUS	埋頭螺栓/M3*P0.5*10L SUS
15-7	5012-03006-200	2	Nyloc Nut	M3*P0.5*6,Multicolored Galvanized	尼龍螺帽/M3*P0.5*6邊 (五彩)
			-----*-----		
15-8	5021-06012-440	2	Flat Washer	φ 6*12*0.6t,SUS	平墊圈/ φ 6*12*0.6t SUS
15-9	5005-06016-440	2	Philip Cap Head Bolt	M6*P1.0*16L,SUS	十字圓頭螺栓/M6*P1.0*16L SUS
15-10	5021-04012-440	2	Flat Washer	φ 4*12*0.8t,SUS,FW04012S	平墊圈/ φ 4*12*0.8t SUS (FW04012S)
15-11	5006-04025-210	2	Self tapping screw	M4*25L,Multicolored Galvanized	自攻螺栓(半圓頭十字)/M4*25L (五彩)
			-----*-----		
15-12	103301-88102-22	1	Front tiller cover	Burgundy	把手前蓋(二合一)/通用 棗紅
15-13	4001151-02	1	Basket Holder	A151-2	籃子掛勾/A151-2 (電著)
15-14	5005-06016-440	2	Philip Cap Head Bolt	M6*P1.0*16L,SUS	十字圓頭螺栓/M6*P1.0*16L SUS
15-15	301307-00	2	Sponge Sticker	75*30*6t	海綿貼紙/75*30*6t
15-16	203205-78700	2	Reflector	PE-35B,Round,Yellow	反光片(黃)/ PE-35B 圓型
			-----*-----		
15-17	203503-88101-50C	1	Cable Cover	Black/Delta Tiller	電線護蓋/通用 黑/環把
15-18	5006-04012-310	1	Self Tapping Screw	M4*12L,Black,888A	自攻螺栓(鑽尾)帶墊/ M4*12L 黑 (888A)
			-----*-----		
15-19	301111-04	2	Instruction sticker	N11-4,F-R	指示貼紙/N11-4 (F-R)

Index NO.	Part NO.	Qty.	Description	Specification	Remark
15-20	301174-05	1	Warning sticker	N74-5	警告貼紙/N74-5
15-21	301174-16	1	Warning Sticker	N74-16	警告貼紙/N74-16
15-22	301123-41	1	Warning Sticker	N23-41	三角型警告貼紙/N23-41
15-23	301123-14	1	Warning sticker	N23-14	三角型警告貼紙/N23-14(大)
15-24	301123-20	1	Warning sticker	N23-20,Front Panel	三角型警告貼紙/N23-20(大)
15-25	301123-35	1	Warning Sticker	N23-35	三角型警告貼紙/N23-35,(配合服務記錄卡)
15-26	301123-08	1	Warning sticker	N23-8,Small	三角型警告貼紙/N23-8 (小)
15-27	301123-30	1	Note Sticker	N23-30 Note for Sleep Mode	三角型注意貼紙/N23-30
15-28	301174-25	1	Warning Sticker	N74-25	警示貼紙/N74-25
15-29	301174-45-A2	1	Warning Sticker	N74-45, A2	警告貼紙/N74-45(A2)
15-30	301123-16	1	Warning sticker	N23-16	三角型警告貼紙/N23-16(大)
15-31	301217-05-A2	1	Trademark Decal, SHOPRIDER	T17-5-A2, 78.5*32	商標貼紙/T17-5-A2 (SHOPRIDER) 78.5*32
15-32	301174-09	1	Warning Sticker	N74-9	警告專利貼紙/N74-9
			-----*-----		
15-A	109901-88101	1	Wicker basket assembly		籃子總成#1/竹藤(SHOPRIDER)(加海綿)
15-33	209916-88101	1	Basket		籃子(竹藤)
15-34	4001357-00	1	Basket stand	Front,A357,SHOPRIDER	籃子掛勾(前)/A357(SHOPRIDER)
15-35	4001358-00	1	Basket stand	Rear,A358	籃子掛勾(後)/A358
15-36	5005-06012-440	2	Screw	M6*P1.0*12L,SUS	十字圓頭螺栓/M6*P1.0*12L SUS
15-37	301307-00	1	Sponginess Sticker	75*30*6t	海綿貼紙/75*30*6t

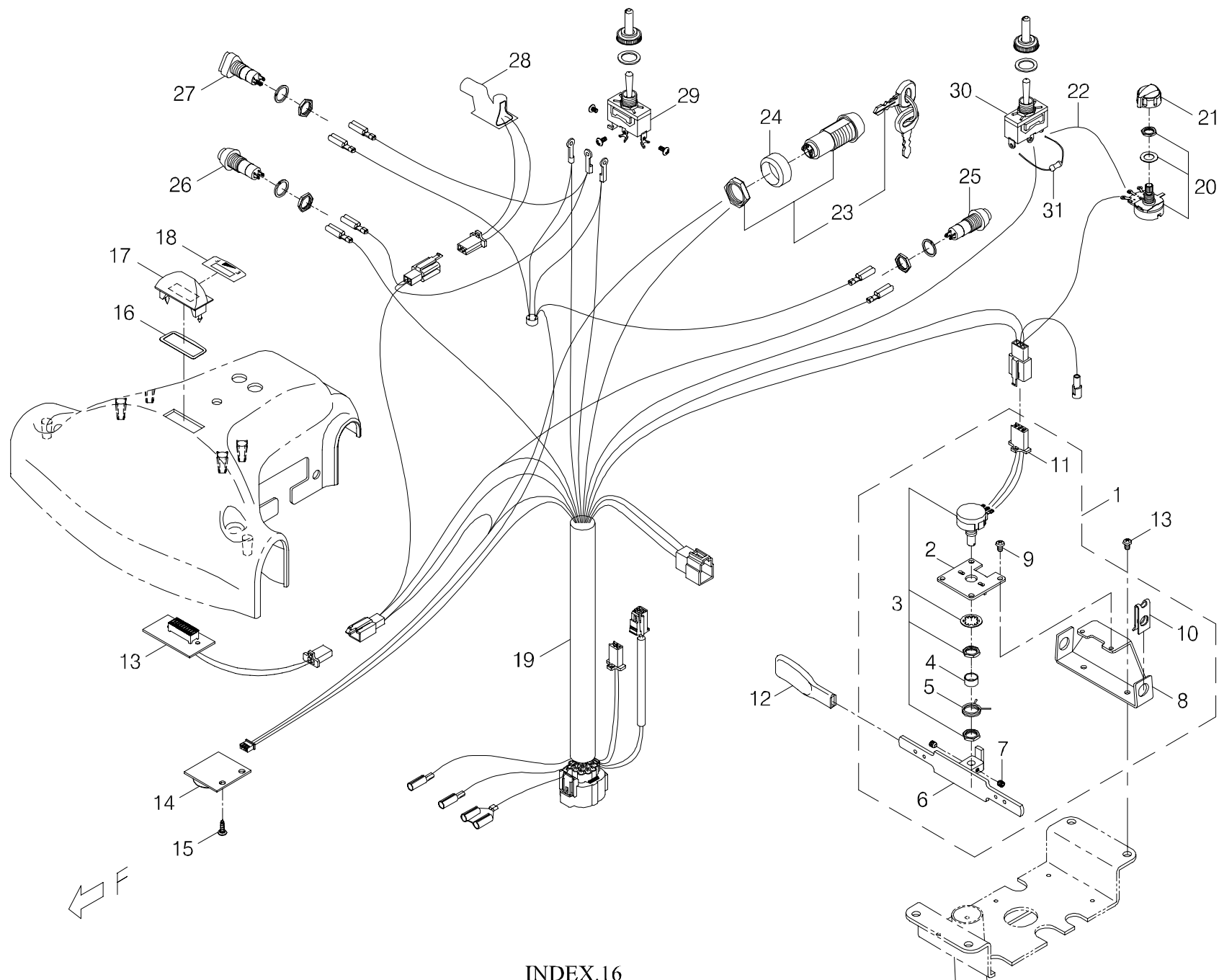


Index NO.	Part NO.	Qty.	Description	Specification	Remark
16	-----		Upper Controller Assembly		上控制總成
16-1	106108-66404-A2	1	POTENTIOMETER,ASBLY	Delta tiller#2,5K#3	搖桿電阻加工/環把#2(5K#3)
16-2	4001144-03	1	Potentiometer Fixing plate	A144-3, ϕ 4.1	搖桿電阻固定片/A144-3 (ϕ 4.1)
16-3	206103-88103-A2	1	Potentiometer	5K#3-A2	搖桿電阻/5K#3-A2(含平墊、螺帽)
16-4	4003198-00	1	Spacer	C198(ϕ 11.3* ϕ 9.5*5.5L)	間隔圈/C198(ϕ 11.3* ϕ 9.5*5.5L)
16-5	206106-88100	1	Spring		扭力彈簧
16-6	206107-88102	1	Throttle lever	#3	加速控制桿/#3
16-7	5004-04004-210	2	Socket Screw	M4*P0.7*4L,Multicolored Galvanized	無頭螺栓/M4*P0.7*4L (五彩)
16-8	4001464-01	1	Potentiometer Fixing plate	A464-1	搖桿電阻固定片/A464-1
16-9	5005-04006-210	2	Philip Screw	M4*P0.7*6L, Multicolor Galvanized	十字圓頭螺栓/M4*P0.7*6L (五彩)
16-10	202921-88100	2	U type plate nut	Close	螺絲固定片/(閉口)
16-11	207010-66402	1	Control cable	KA-889-K6	上控制線/KA-889-K6
-----	505201-00940-50	1	cable Tie	CV-094B	束帶/CV-094B
			-----*-----		
16-12	206122-88102	2	Extension, thottle lever	PVC,Black#1	加速桿加長片/PVC (黑)#1,長
			-----*-----		
16-13	206114-66103-A4	1	PCB for Battery Gauge	AC LED-B1-J, A4	電量指示器PC板/AC LED-B1-J(有端子)(A4)
16-14	206217-66801-A2	1	Horn PCB	PH-27,A2	音效器/PH-27(A2)
16-15	5006-04012-213	2	Self tapping screw	M4*12L,Multicolored Galvanized	自攻螺栓(TP丸頭)/M4*12L (五彩)尖尾
16-16	206128-80500	1	Gasket	□34.7*12.1*0.3T	指示燈座墊片/ □34.7*12.1*0.3T
16-17	206126-88100	1	Indicator lamp	Black	指示燈座/黑色
16-18	301192-00	1	Decal	N92,E,-,F	電量指示貼紙/N92 (E - F)
			-----*-----		
16-19	207010-77802	1	Upper Control Cable	RA-888-K3,Delta tiller	上控制線/RA-888-K3(環把)


Index NO.	Part NO.	Qty.	Description	Specification	Remark
16-20	206109-88100-A2	1	Variable resistor	B25K(A2)	可變電阻/B25K(含平墊、螺帽)A2
16-21	206113-88101-A2	1	Speed Dial Knob	CF-X4-T	速率調整鈕/CF-X4-T(φ 19.8)(A2)
16-22	207012-66200	1	Jump Wire	KA-888-G5	跳線/KA-888-G5
			-----*		
16-23	106106-88103-A3	1	ASM,IGNITION W/KEYS	2-Way Key,4A 125VAC 2Amp 8212,A3	鑰匙組總成(黃色)/雙向,4A 125VAC 2Amp 8212(A3)
			-----*		
16-24	4003260-00	1	Spacer	C260(φ 26*φ 20*12L)	鑰匙間隔圈/C260(φ 26*φ 20*12L)
			-----*		
16-25	206105-88100-A2	1	Horn Switch	Plain Yellow Button	喇叭按鈕(不銹鋼彈簧)/黃色(含螺帽、彈墊)A2
16-26	208201-88100-A2	1	Light Switch	Blue	藍色按鈕(不銹鋼彈簧)/二段式(含螺帽、彈墊)A2
16-27	208202-88103-A2	1	Hazard Warning Switch	2pin,Red	三角型按鈕(不銹鋼彈簧)/2pin 紅色(含螺帽、彈墊)A2
16-28	208204-88101-A2	1	Flasher	ASIC,24V,2A	閃光器/ASIC,24V/2A(A2)
16-29	208203-88100	1	Indicator Switch	3P,ON-OFF-ON	搖頭開關/ 3P ON-OFF-ON (含防水套、墊片)
16-30	208203-88101	1	Toggle switch	3P,ON-ON	搖頭開關/3P ON-ON(含防水套、墊片)
16-31	206120-89501	1	Resistor	10K 1/4W	電阻(精密)/10K 1/4W
			-----*		
-----	505201-00940-50	6	Cable tie	CV-094B	束帶/CV-094B
-----	505201-01200-50	3	Cable tie	CV-120B	束帶/CV-120B
-----	505201-02000-50	1	Cable tie	CV-200B	束帶/CV-200B

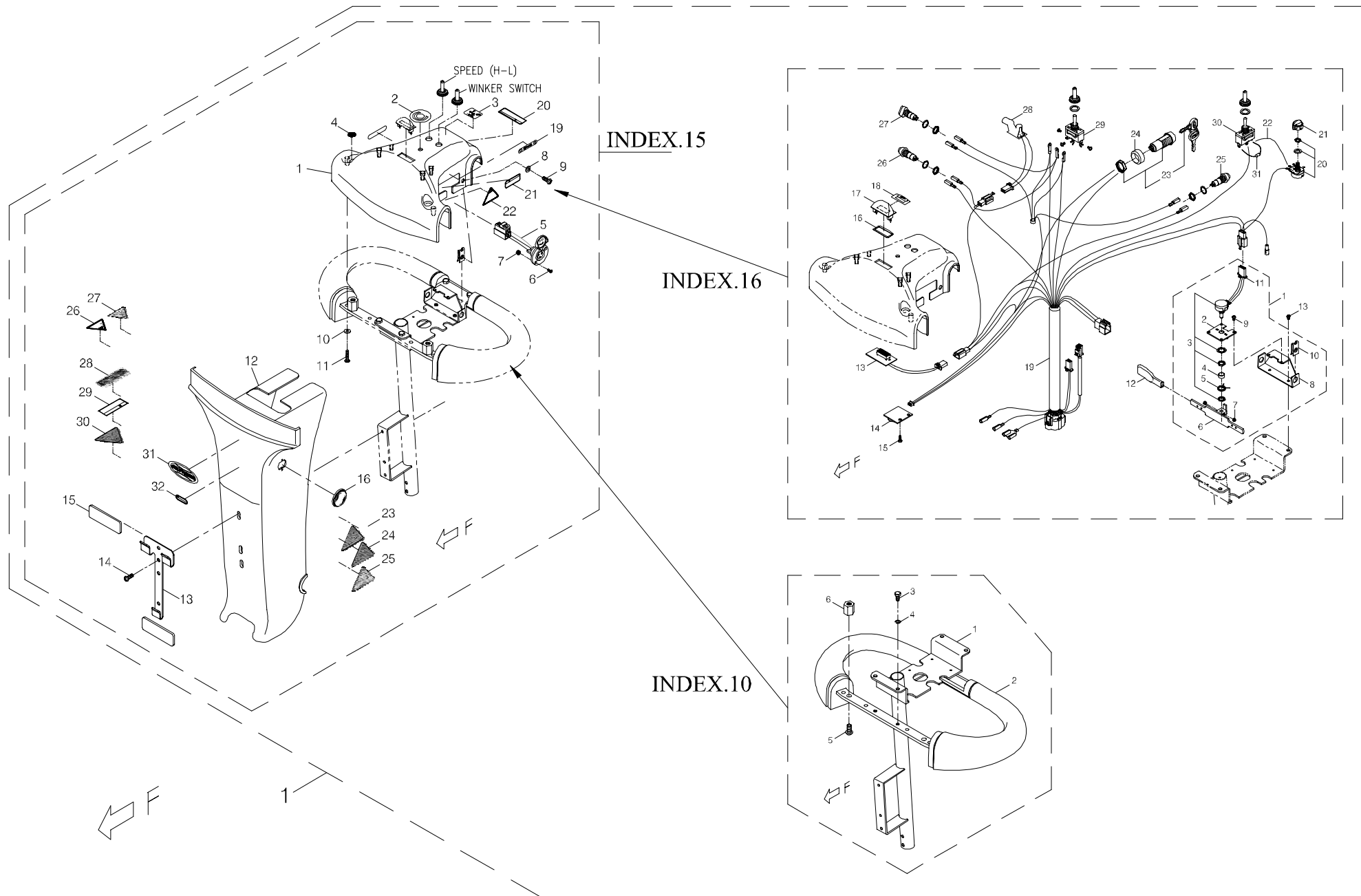


INDEX.17

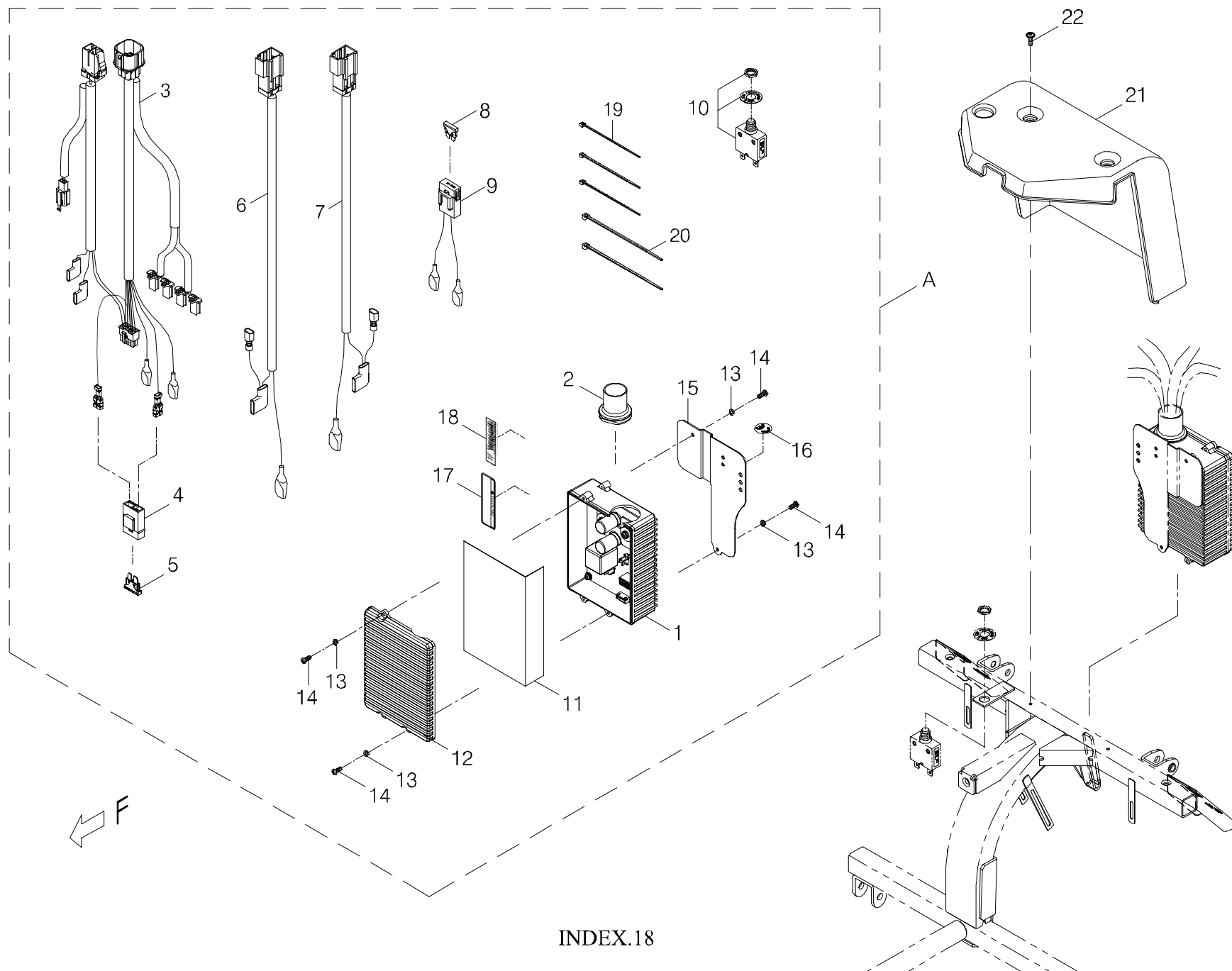


Index NO.	Part NO.	Qty.	Description	Specification	Remark
18-A	106202-88901-A3	1	Main Control Box Ass'y(w/ Bracket etc.)	AC2-110, 889SL/SEL w/ C or Delta Tiller	下控制總成(C、環把)含固定片配/(AC2-110)889SL/SEL
18-1	206221-88900-A2	1	Controller	AC2-110, w/ Case	控制器/AC2-110(含盒)(A2)
18-2	206210-88103	1	Protection	AC2-110/150	護套(O)型環/AC2-110/150
18-3	207005-89815-A2	1	Main control cable	NA-889-J20-A2	主控制線/NA-889-J20-A2
18-4	206206-88101	1	Fuse housing		保險絲座/盒式
18-5	206901-88100	1	Fuse	2A	保險絲 (盒式)/ 2A
18-6	207003-67204	1	Power Cable	+,RA-889-D4,GR889/TE-9	電源線(正)/RA-889-D4(GR889/TE-9)
18-7	207004-67204	1	Power Cable	-,RA-889-E4,GR889/TE-9	電源線(負)/RA-889-E4(GR889/TE-9)
18-8	206901-88113	1	Blade Fuse	UL 15A	保險絲 (盒式)/UL 15A
18-9	207014-88105	1	Connection Cable	KA-888-L8	連接線/KA-888-L8
18-10	206207-88102	1	Circuit Breaker	50A	安全開關/50A(含平墊、螺帽)
18-11	302002-00	1	Insulator	152*129m/m*0.25t	絕緣膠片/152*129m/m*0.25t
18-12	206209-68500	1	Control box cover	AC2-110/150	下控制盒蓋/AC2-110/150
18-13	5022-04007-210	4	Spring washer	φ 4*7*1.25t,Multicolored Galvanized	彈簧墊圈/ φ 4*7*1.25t (五彩)
18-14	5005-04010-210	4	Philip Screw	M4*P0.7*10L,Multicolored Galvanized	十字圓頭螺栓/M4*P0.7*10L (五彩)
18-15	4001383-08-A2C	1	Bracket for Controller	A383-8(A2)	控制固定片/A383-8(A2)
18-16	301166-02	1	CE Mark Label	N66-2, White Sign	CE貼紙/N66-2 透白
18-17	301174-17	1	Warning sticker	N74-17	警告貼紙/N74-17
18-18	-----	REF(1)		DECAL	條碼貼紙
18-19	505201-00940-50	2	cable Tie	CV-094B	束帶/CV-094B
18-20	505201-01500-50	3	cable Tie	CV-150B	束帶/CV-150B
			-----*-----		
18-21	203510-88801-A2C	1	Rear Decoration Cover	for 888SEL/889SEL w/ AC2, A2	後飾蓋(射出)/888SEL/889SEL(A2)(AC控制)
18-22	5006-04012-310	2	Self Tapping Screw	M4*12L,Black,888A	自攻螺栓(鑽尾)帶墊/M4*12L 黑 (888A)

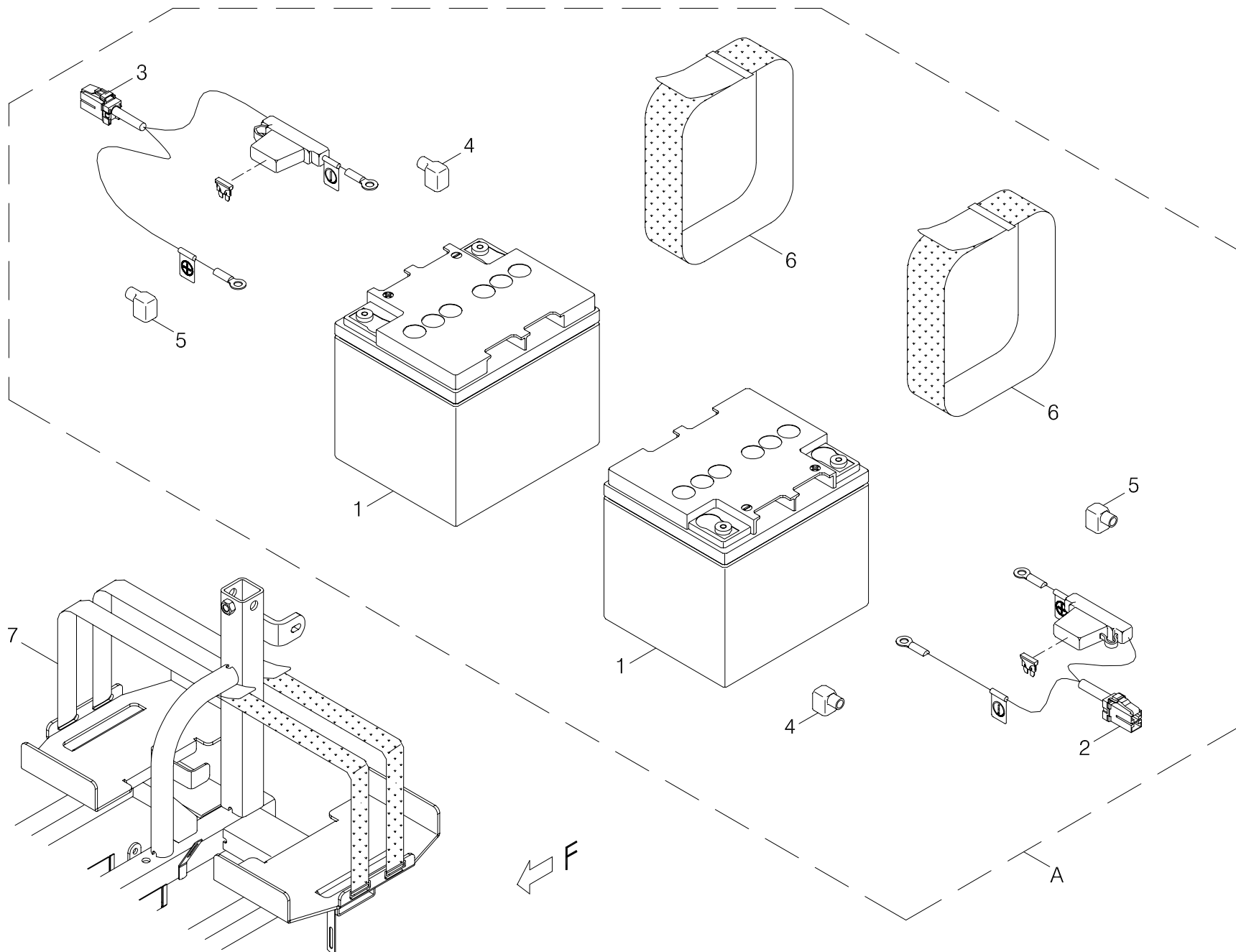

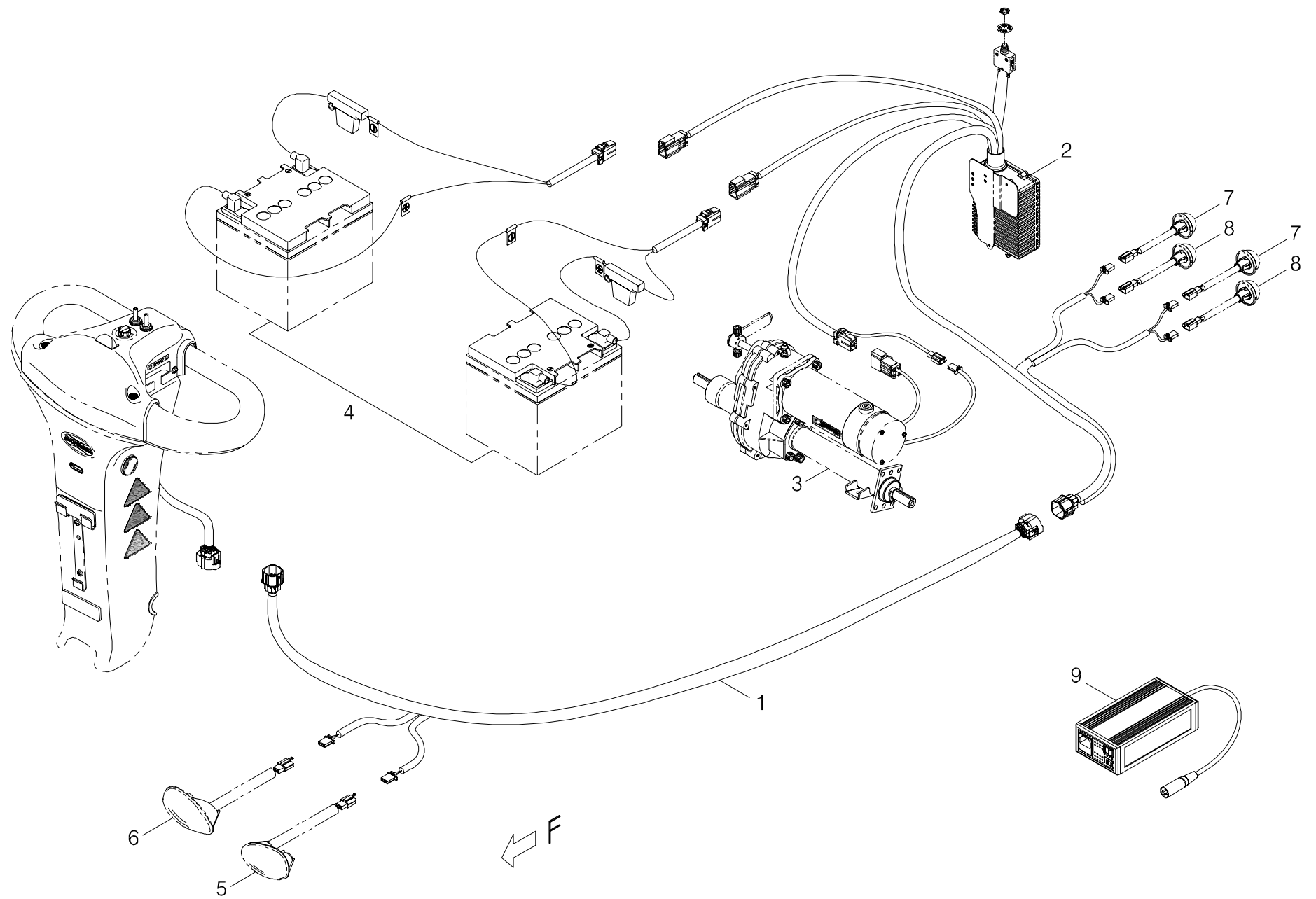

Index NO.	Part NO.	Qty.	Description	Specification	Remark
20	-----		Control and battery wiring illustration		控制及電瓶導線總成
20-1	207011-67204	1	Control Connection Cable	RA-889X-K3	控制連接線/RA-889X-K3
			-----*		
20-2	-----	REF(1)	Main Control Box Ass'y(w/ Bracket etc.)		下控制總成(C、環把)含固定片配/(AC2-110)889SL/SEL
			-----*		
20-3	-----	REF(1)	Drive-train Assembly		馬達剎車變速箱總成/9MNF-2B+(#6)+889SEL NTBГ
			-----*		
20-4	-----	REF(1)	SLA Batteries w/ Battery Cables		加電瓶模組(UPG)/G424M (50Ah)(A2)
			-----*		
20-5	-----	REF(1)	Head light		前燈 (左)/CAT-L FUAN 0053
20-6	-----	REF(1)	Head light		前燈 (右)/CAT-R FUAN 0053
			-----*		
20-7	-----	REF(2)	Winker/Indicator Assembly		方向燈總成/圓型/插頭 (黃色)FUAN 0054(A2)
20-8	-----	REF(2)	Taillight Set		尾燈總成/圓型/插頭 (紅色)FUAN 0054
			-----*		
20-10	109201-88125	1	Off-board Charger	4A, for GEL Type SLA Battery	充電機(鉛酸/膠狀電瓶共用)/4A
20-11	209201-88106	1	Power cord	Australia,ZGDZ1	充電機電源線/澳洲 (ZGDZ1)

Wiring Diagram for TE-889SL

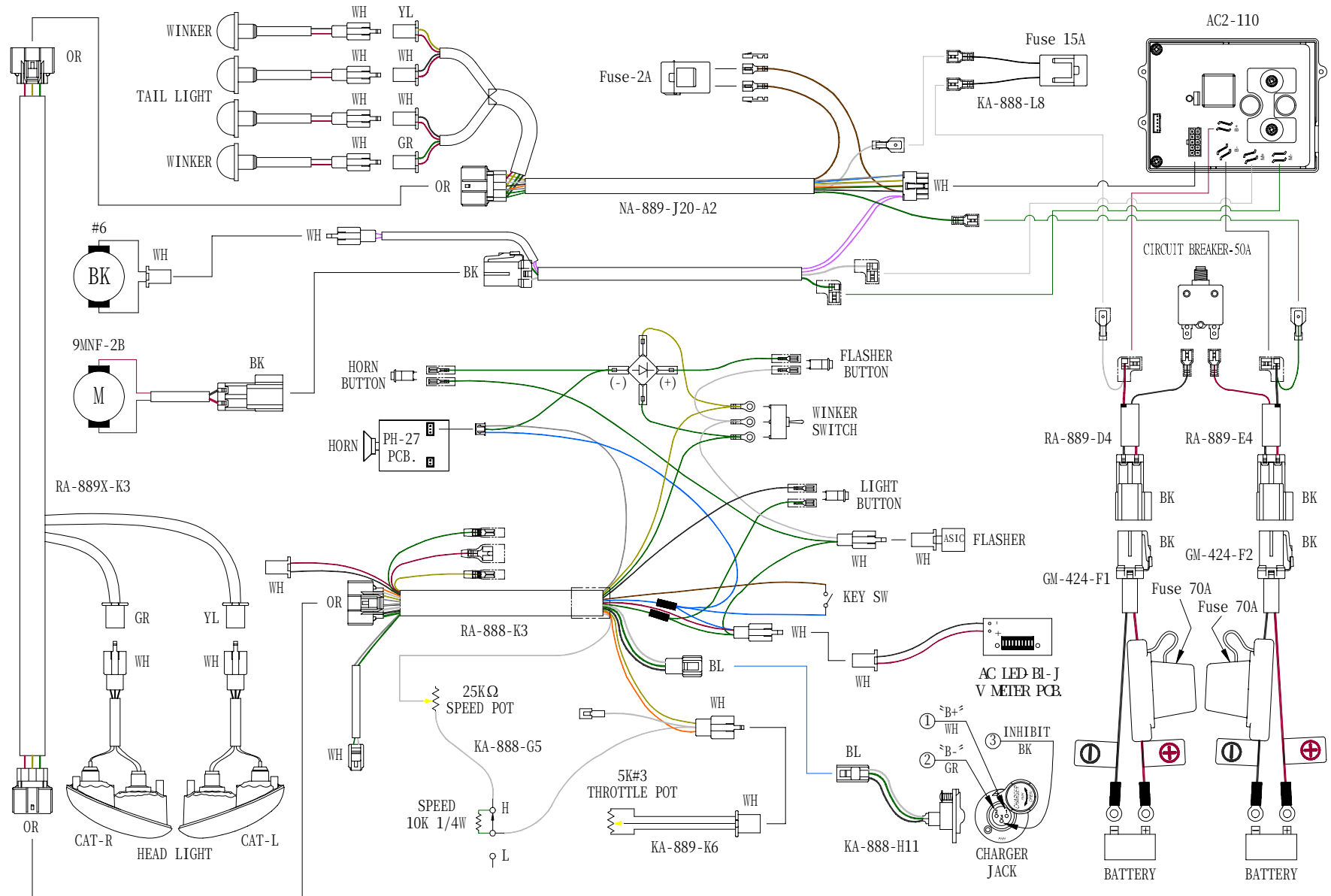

DWG NO. : AC2-110-010
EDITION : A4